

# User Manual

Version 3.7

## EzScan-i



# vatech

Document	Software Version	Date (m.d.y)
EzScan User Manual IOS-FPM-71-007_EN_AA	3.4	12.23.2021
	3.5	02.07.2022
	3.6	07.07.2022
	3.7	10.25.2022

Introducing the EzScan .....7

1.1 Introduction .....7

1.2 What’s new in this version.....7

1.3 Indications supported .....7

1.4 Certification and compliance .....7

1.5 Regulatory .....8

1.6 Intended Use .....8

1.7 Classifications .....8

2. Components Overview .....9

2.1 Hardware Components.....9

2.2 Software Components .....9

2.3 System Requirements and Specifications .....10

2.4 Software output and design software compatibility .....11

2.5 Computer Requirements .....11

2.6 Environment Conditions .....13

2.7 Power Input .....13

2.8 Reusable Tips .....13

2.9 Scanner Base and Handpiece .....13

2.10 Calibration .....14

3. Installing and connecting the EzScan .....15

3.1 Wall Mount Installation Instructions (Optional) .....15

3.2 Connecting the EzScan.....16

3.3 Configuring Windows High Performance Graphics Mode .....18

4. Getting Started With EzScan-i .....20

4.1 Device Registration .....20

4.2 Accessing the EzScan-i Homepage.....22

4.3 Customizing the EzScan-i Admin User Account .....23

4.4 Adding a New User Account .....24

4.5 Accessing Your EzScan-i User Account.....25

4.6 Linking a User Account to the EzScanCloud.....25

5. Configuring EzScan-i Settings .....26

Accessing the Settings Menu .....26

5.1 General Settings .....26

5.2 Data Settings.....28

5.3 Sound Settings.....31

5.4 User Settings.....32

5.5 Upload Settings .....	32
5.6 Case Setup Settings .....	33
5.7 Lab Sheet Settings .....	40
5.8 3D Settings.....	40

## 6. Managing Cases in EzScan-i .....43

6.1 Managing Patient Profiles .....	43
6.2 Managing Patient Cases .....	45
6.3 Setting Up a Case in EzScan-i .....	47

## 7. Scanning with EzScan.....51

7.1 Before You Scan.....	51
7.2 Using the Scan Tip .....	52
For ease of use, hold the EzScan handpiece as shown.....	52
7.3 An Example Scan Workflow Menu .....	53
7.4 Useful Scanning Tips.....	54
7.5 Accessing the Scan Workflow .....	56
7.6 Scanning the Maxillary Arch .....	56
7.7 Scanning the Mandibular Arch .....	56
7.8 Recommended Scanpath Strategy .....	57
7.9 Scan Memory Use.....	59
7.10 Scan Live View .....	60
7.11 Using Scan Tools .....	60
7.12 Using the Quality Map ¶.....	61
Disable/Enable Captured Color ¶.....	62
7.13 Using EzScan Live Scan Tools ¶.....	64
Undercuts: Auto-Detect ¶.....	67
7.14 Carrying Out A Bite Alignment.....	71
Accessing Bite Alignment in the Scan Workflow .....	71
Changing between automatic and manual mode .....	71
Using audio-guidance.....	71
7.15 Carrying out an Automatic Bite Alignment.....	72
Recommended Scanpath Strategy .....	72
Bite-alignment using a single bite segment .....	73
7.16 Carrying out a Manual Bite Alignment .....	74
7.17 Auto-Realignment.....	77
7.18 Using Bite Alignment Tools.....	77

## 8. Finalizing A Patient Case .....79

8.1 Finalizing Your Order .....	79
8.2 Case Finalization Options.....	79
8.3 Using Case Finalization Tools.....	81
8.4 Using the Margin Line Tool.....	83
Modifying a Margin Line ¶.....	84

8.5 Using the Model Closing Tool ¶	85
8.6 Case Review: Reviewing 2D images	87
8.7 Case Review: Reviewing 3D Models	87
8.8 Check And Send Your Order	88
Verify Patient & Restoration details	88
Verify Export settings	88
Verify Lab Order Details	88
Select Default Lab	89
Verify Order Form Details	89
Verify Case Status	90
Resending a Modified Order Form	90

## 9. Communicating With Labs .....92

9.1 Linking an EzScan-i account to EzScanCloud	92
9.2 Creating An EzScanCloud User Account (Clinic)	93
9.3 Overview of the EzScanCloud Dashboard	94
9.4 Configuring EzScanCloud User Settings	95
9.5 Editing your EzScanCloud Profile	95
9.6 Configuring View & Notification Settings	96
9.7 Configuring Connections with Labs	97
Selecting Default Upload Settings in EzScan-i	98
9.8 Managing Cases in EzScanCloud	99
9.9 Managing Case Status in EzScanCloud	100
Displaying Cases by Status	100
Filtering Cases	100
Changing Case Status	101
Case Status Descriptions	101
9.10 Displaying 3D Views in EzScanCloud	102
9.11 Downloading Case Files in EzScanCloud	102
Downloading specific case files	102
Downloading all case files	102
Case File Details	103
9.12 Displaying a Case Order Form in EzScanCloud	103
9.13 Adding a Comment to a Case	103
9.14 Contacting Support via EzScanCloud	105

## 10. Maintenance .....106

10.1 Cleaning the Handpiece	106
Recommended and approved surface disinfectants:	106
10.2 Cleaning and Sterilizing Tips	106
10.3 Disposal	108
10.4 Calibration	108

## 11. Safety Guidelines and Warnings .....109

11.1	Warnings and Symbols.....	109
11.2	General Guidelines .....	109
11.3	General Warnings .....	110
11.4	Mechanical Hazards.....	110
11.5	Electrical Safety .....	111
11.6	Eye Safety .....	112
11.7	Hygiene.....	112
11.8	Precautions During Systems Operation .....	112
11.9	EzScan Performance .....	113
11.10	Accessories .....	114
11.11	EMC Guidance and Declaration .....	114
11.12	Symbols on the EzScan and TransportCase .....	118

## 12. Support, Warranty & RepairService .....120

12.1	Support.....	120
12.2	Standard Warranty .....	120
12.3	Repair Service .....	121
12.4	Out of Warranty RepairService.....	121

# Introducing the EzScan

## 1.1 Introduction

Thank you for purchasing the EzScan solution from Vatech. The EzScan solution is designed and developed to produce high-quality digital intraoral scans or models, for dental restoration or analysis.

The EzScan solution is designed with you as a dentist and your patient in mind. The scanner is lightweight, small and easy to use, enabling fast, accurate scanning and great patient experience. The Case management application provides efficient and customizable order management, for easy and intuitive use. Enjoy your new intraoral scanner solution!

## 1.2 What's new in this version

This document update incorporates new features and improvements added to the EzScan-i 3.7 software release.

For more information consult: EzScan-i 3.7 Release Notes.



## 1.3 Indications supported

The data sets from the EzScan can be used for the following indications.

- Conventional crowns
- Anatomic crowns
- Copings
- Provisional crowns
- Anatomical pontics
- Reduced pontics
- Provisional pontics
- Inlays/Onlays
- Implant abutments
- 3-unit implant bridges
- Up to 5-unit bridges
- Orthodontic aligners
- Nightguards
- Splints
- Retainers
- Bleach trays
- Sleep appliances
- ...

\* Verify with your dental lab or service provider about their capabilities to produce particular indications.

## 1.4 Certification and compliance

The system has been tested and conforms to the following standards:

- IEC 60601-1, Medical electrical equipment - Part 1: General requirements for basic safety and essential performance
- IEC 60601-1-2, Medical electrical equipment - Part 1-2: General

requirements for basic safety and essential performance - Collateral  
Standard: Electromagnetic disturbances

- Requirements and tests:

- IEC 62471, Photobiological safety of lamps and lamp systems

## 1.5 Regulatory

The EzScan is manufactured and marketed in accordance with US FDA Regulations and EU Medical Device Regulation 2017/745.

## 1.6 Intended Use

The EzScan is an optical impression system. It is used to record the topographical characteristics of teeth, dental impressions, or stone models for use in the computer-aided design (CAD) and computer-aided manufacturing (CAM) of dental restorative prosthetic devices.



**CAUTION:** Rx only - Federal law restricts this device to be sold by or on the order of a Dentist.



**WARNING:** Unintended use of the system can result in physical injury to the patient and operator, and damage to the system.

## 1.7 Classifications

**The EzScan system has the following classifications**

- Protection against electrical shock: Type B Applied Part
- Protection against harmful ingress of water: Ordinary equipment (IPX0)
- Safety of application in the presence of a flammable anesthetic material with air or with oxygen or nitrous oxide: Equipment not suitable for use in the presence of a flammable anesthetic mixture with air or with oxygen or nitrous oxide.




# 2. Components Overview

The EzScan system is composed of hardware and software components:

## 2.1 Hardware Components

The EzScan system is composed of the following hardware components\*:

- 1. EzScan Scanner
- 2. EzScan base
- 3. Removable autoclavable tips
- 4. USB 3.0 cable
- 5. AC/DC power adapter

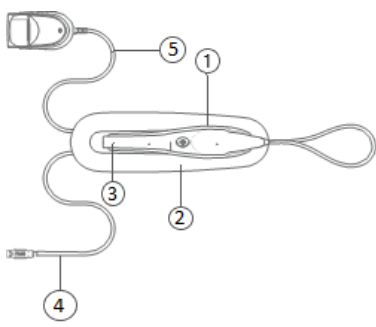

Hardware Component	Part Number
EzScan System (Includes 3 Tips)	V2810110
USB 3.0 Cable	IOS-CP-00-043
AC/DC Power Adapter	IOS-CP-00-088

**Save the Box:** It is highly recommended that you store the box in a safe place and do not dispose of it. The box is optimal for any necessary transportation or shipment of the EzScan.

## 2.2 Software Components

The EzScan system comprises the following software components:

- EzScan-i : 3D Scanning and Case Management software.
- EzScanCloud : Dedicated cloud platform for ordering and communicating with labs

## 2.3 System Requirements and Specifications

### Scanner Specifications

Scanner Type	Hand-held (chairside) scanner that creates optical impressions for dental restorations.
Design	Compact, lightweight, ergonomic – designed to be operated with little physical effort.
Dimensions Base	Size: L 306mm, W 98mm, H 72mm
Dimensions Scanner	Weight: 150 grams
	Size: L 256mm, W 43mm, H 43mm
	Cable length (scanner to base): 2m
Power Requirement	DC 5.0V / 4A (Power supply included)
Scanner Tip	Reusable up to 250 times, sterilize using steam autoclave
Heating Element	Ventilated. Prevents formation of fog on optics
Acquisition Method/Imaging technology	Hybrid technology: active stereo imaging and structured light
Sensor technology	CMOS
Color Scanning	24-bit (8-bit per channel)
Scanning frequency	25-30 FPS
Imaging field-of-view	12mm x 14mm
Light sources	High-power LEDs

### Scanning Process

Tooth Preparation	No powder or spray required
Scanning Principle	Continuously scanning and accumulating (stitching) depth and color data
Distance Scanner - Tooth	-1mm – 19mm
Possible contact duration by operator	<10 min. Note: May vary with hardware configuration

Operator accessible part	Handpiece
Possible contact duration by patient	$t \leq 10 \text{ min}$
Patient accessible part (Type B Applied Part)	Tip (autoclavable)
Computer – Scanner Interface	USB 3.0

## 2.4 Software output and design software compatibility

Output File Format	STL, PLY, OBJ
Compatibility with CAD/CAM Systems	Open Architecture Output format STL, PLY, OBJ Compatible with most Dental CAD systems

## 2.5 Computer Requirements

### Minimum Software Requirements

Operating System	Windows 10 (Excluding Windows 10 S, now defunct) Administrative rights required
Disk Space	100 GB or greater of free disk space
Ports	At least 1 x USB 3.0 port (SuperSpeed)
Nvidia Driver	Nvidia Studio driver version 471.68 is currently required. Nvidia gamer-ready driver should not be used with the EzScan-i software.
NVIDIA GPU operation mode	The PC must be able to ensure that the Nvidia GPU is the only GPU activated on the PC and that any Intel integrated GPU is disabled. (This is usually achieved using high level configuration tools or BIOS commands).  <b>Warning:</b> Certain models of computer of the following brands - Acer, MSI, HP,... - provide no option to only have the Nvidia GPU active. This may affect performance: low FPS or freeze during scan.
Screen resolution	<b>Full HD (1920 x 1080) with DPI 100%</b> <b>NOTE:</b> The use of 4K (3840 x 2160 pixels) or Ultra-Wide (3440 x 1440 pixels) resolutions is also possible,

	however the impact on performance has not been quantified at this stage.
--	--

### Software Configuration Recommendations

Windows automatic updates	We recommend deactivating all Windows automatic updates (except for security updates).
Nvidia driver automatic updates	Nvidia driver automatic updates should be disabled.
Windows Battery Settings	On laptops, the battery setting in Windows should be configured to <b>high performance mode only</b> , with <u>no battery saving option</u> .

### Minimum Hardware Requirements

CPU Type	Intel 10, 11 and 12 generation. Intel i7 or i9 – 4 Cores give best performance.
CPU Clock	2.8 GHz clock or greater
Memory	16 GB of RAM or greater (DDR4 or better)
Graphics Card Memory	6GB of RAM minimum on the graphics card are needed. Below this, the software will not launch: an error message will inform you that the minimum requirement is not reached.
GPU	Quadro RTX3000, RTX4000 and above for laptop and desktop RTX2070 for laptop and desktop RTX2080 for laptop and desktop RTX2080TI for desktop RTX3070 for laptop and desktop RTX3080 for laptop and desktop RTX3090 for desktop



**WARNING: AMD GPUs are NOT guaranteed compatible with EzScan.**

Not meeting minimum hardware requirements will affect the performance of the scanner.

## 2.6 Environment Conditions

Operating Temperature	10°C to 40°C
Operating Relative Humidity	10% to 80% (non-condensing)
Storage Temperature	- 20°C to 60°C
Storage Relative Humidity	10% to 80% (non-condensing) Indoor use only
Installation Category	1
Pollution Degree	2
Ingress of Liquids	IPX0
Protective Class	Class IIIb
Overvoltage category	II per IEC 60664-1
Max. working condition	Continuous cycles with image capture and transmission from/to Notebook or non-medical grade PC.
Other possible accessories (IEC60601-1 3rd, Cl. 16)	Notebook with AC/DC Adapter.
Equipment Maintenance	No user maintenance is required, and no user service is allowed. Please contact technical support in case of problem.
Cleaning	Do not try to clean the inside of the device Refer to <b>section 9.1: Cleaning the Handpiece</b> for cleaning and sterilization.

## 2.7 Power Input

The power adapter input is 5V DC, 100-240V AC, 50-60Hz.

## 2.8 Reusable Tips

Scanner tip is autoclavable up to 250 times in a steam autoclave when used with 132°C/134°C 4 mins cycle or 121°C 45 min cycle.

See below: section 10.2 **Cleaning and Sterilizing Tips**.

## 2.9 Scanner Base and Handpiece

The scanner body consists of the Docking Base and Handpiece, which are connected by a flexible, non-detachable cable.

## 2.10 Calibration


The EzScan™ intraoral scanner is factory calibrated. In the case of calibration issues due to transport, please contact your reseller or Vatech support technician.

## 3. Installing and connecting the EzScan

### 3.1 Wall Mount Installation Instructions (Optional)

When wall mounting the scanner's base, be careful to respect measurements indicated to drill holes for the wall mount.

- It is recommended to use wall anchors and threaded screws with an 8mm (5/16th inch) head diameter.
- A 4-5mm (3/16th in) distance between wall and bottom screw head is recommended.

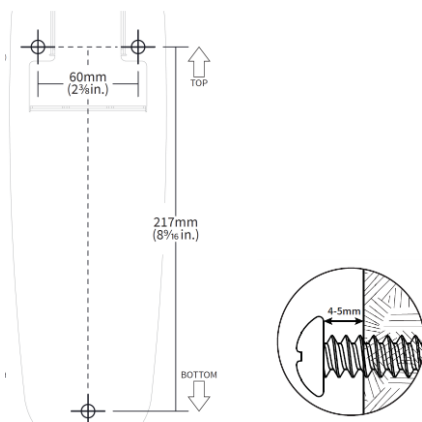


**Caution:** If the docking base is not properly installed there is a possibility of the base falling and damaging the scanner.



**IMPORTANT:** Vatech is not responsible for accidents or damage caused in the event the scanner falls.

## 3.2 Connecting the EzScan

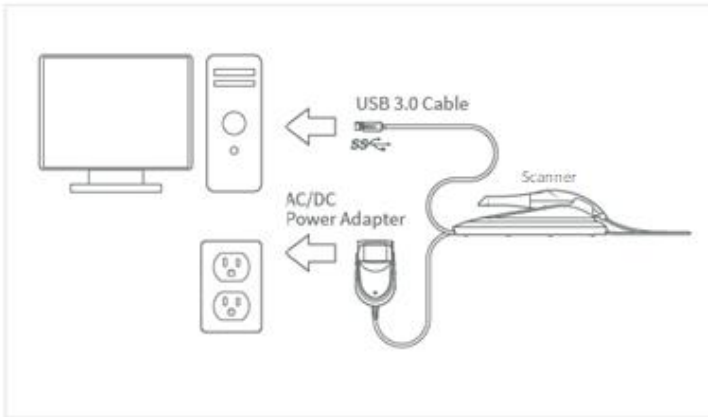

To install and connect the EzScan Scanner:

- STEP 1.** Place the docking base on a flat, stable surface and place the EzScan handpiece securely on the base.
- STEP 2.** Connect the AC/DC power adapter cable to the docking base (the connector socket is located underneath the base of the scanner).
- STEP 3.** Connect the provided USB 3.0 cable to the docking base (the connector socket is located underneath the base of the scanner).



**WARNING:** Make sure you use the 5.0V 4A power adapter provided. Failure to do so may result in damage to the scanning device.



**WARNING:** Using a USB cable other than the one provided may result in system malfunction or reduced performance.



**STEP 4.** Connect the other end of the USB 3.0 cable to the computer.



**IMPORTANT:** Make sure to use a USB port that is compatible with USB 3.0 (SuperSpeed), usually indicated by this symbol: not doing so may result in system malfunction or reduced performance.



**IMPORTANT:** When using a desktop computer, it is strongly recommended to plug the USB cable to a USB port located at the back of the computer; not doing so may result in system malfunction or reduced performance.

**DO NOT** plug the USB cable into an intermediate hub.

**STEP 5.** Connect the adapter block provided to a power outlet.




**IMPORTANT:** If using a laptop computer, make sure the power supply is connected to a power outlet and not running on battery power. Failure to do so will mean that the scanner will not have sufficient power to produce images.

### **IMPORTANT: Connect to Power Supply before Scanning!**


If your laptop computer is not properly connected to a power outlet, the following message will appear:

In this case, please connect your laptop to the power supply **before proceeding to use to scanner.**

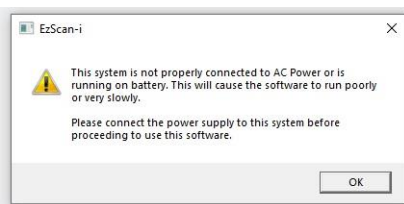


### 3.3 Configuring Windows High Performance Graphics Mode

On laptop computers, battery settings in Windows should be configured to high performance mode only, with no battery saving option (Settings/System/Battery).

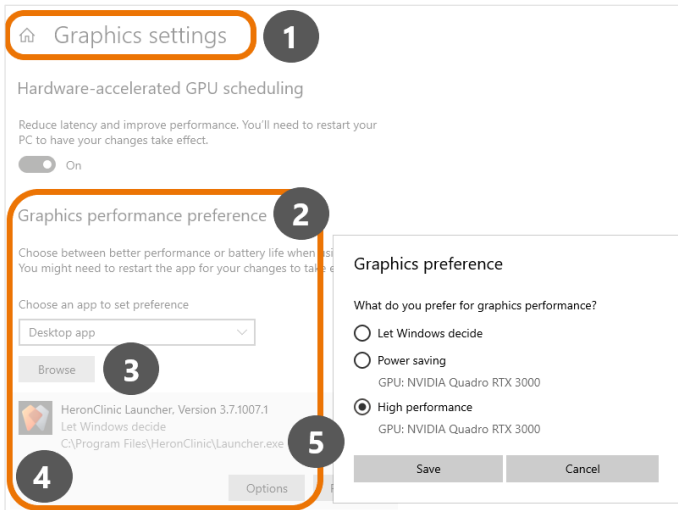

To select High Performance mode for the HeronClinic application:

1. In Windows **Graphic settings**: click to activate **Hardware-accelerated GPU scheduling**
2. In **Graphics performance preference**: select **Desktop app**
3. Click **Browse** and select the **HeronClinic** app as shown (**Launcher.exe**):

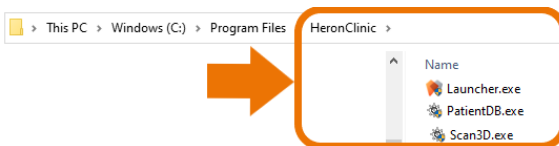

4. In **Options**, select **High performance**, and click **Save**.
5. Repeat this for the **HeronClinic PatientDB.exe** and **Scan3D.exe** executable files.
6. Restart your PC to apply changes.



**IMPORTANT:** On laptop computers, battery settings in Windows should be configured to high performance mode only, with no battery saving option (Settings/System/Battery).

## **WARNING: Unsuitable installation sites**




**WARNING:** Unsuitable installation sites:

- Locations with excessive humidity or dust
- Locations subject to high temperature
- Locations subject to shaking or vibration
- Locations exposed to considerable electrical or magnetic noise, or other forms of electromagnetic energy

## 4. Getting Started With EzScan-i

Once you have connected the EzScan, you are ready to launch the EzScan-i software installed on your computer. The EzScan-i software is launched by selecting the **IO Scanner** option in **EzDent-i**.

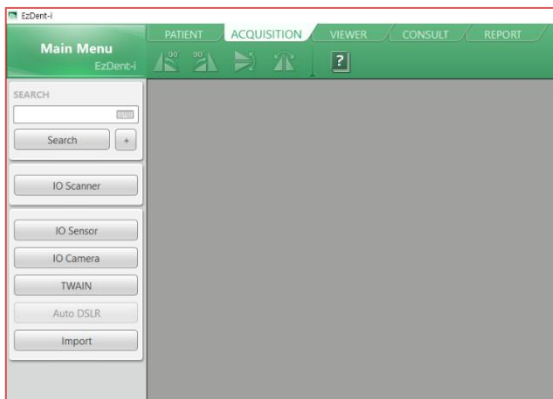

### 4.1 Device Registration

When you launch the EzScan-i software for the first time, you will be invited to register your EzScan device:

In the Device **Registration** dialog box:

- STEP 1.** Enter your personal contact details.
- STEP 2.** Copy/paste the scanner serial number from General settings
- STEP 3.** Tick the consent box to agree to the storage and processing of your data.
- STEP 4.** Click to **Submit** device registration information.


When you submit the device registration form, the EzScan-i interface will open directly in the **Case Setup** page.



**WARNING:** It will not be possible to perform new scans or export existing scans if the device has not been registered on the PC.

## Case Setup

The Case Setup icon is highlighted in the left-hand menu.

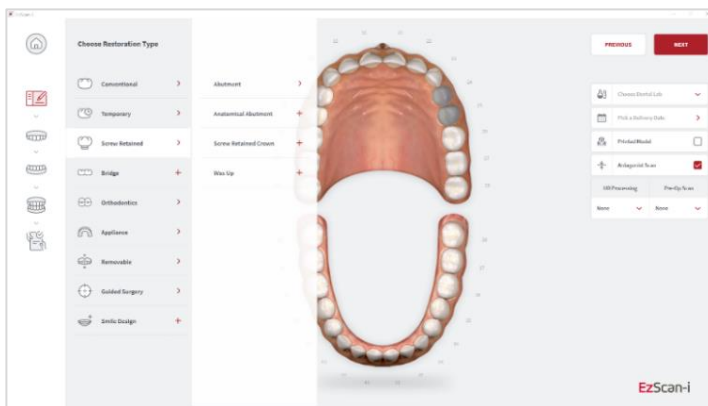


The Case Setup page enables you to select restorations, and to enter restoration and order form details.

For more information see below: section 6.3 **Setting Up a Case in EzScan-i**


## 4.2 Accessing the EzScan-i Homepage

To go to the EzScan-i homepage, click on the **Home** icon in the top left corner of the **Case Setup** window.

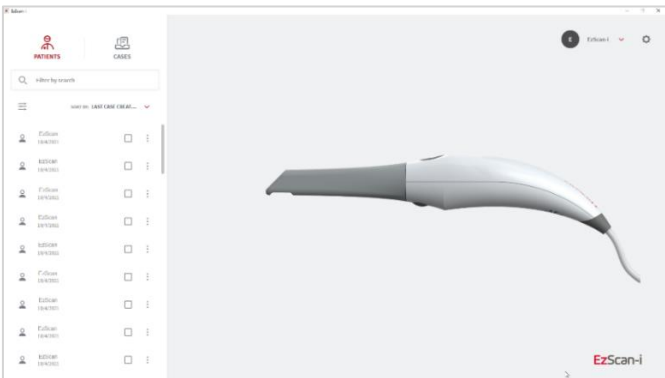

### Patients/Cases View

In the EzScan-i homepage, you can toggle between **Patients** and **Cases** views, and Filter or search desired Patients/Cases.

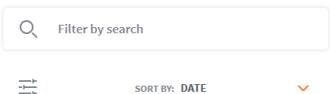


### Filter/Search

You can search by term or Click on the Filter icon to open the Filters dialog box.



You can display a list of Patients by:

- **Gender**
- **Case Creation Date** (or range of dates)
- **Date of Birth**

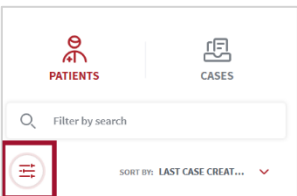


An active filter is indicated as shown above.

### Select A User Start Page Default View

To select a preferred **Default View** for the **User Start Page**:


- STEP 1.** Click the System icon located in the top right-hand corner of the User Start Screen, to access the Settings interface.
- STEP 2.** In the left-hand **Settings** menu select **General**.
- STEP 3.** Click **Preferred Start Page**, and in the drop-down menu select **Recent Patients** or **Recent Cases**.

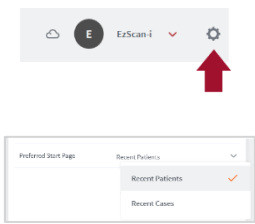

## 4.3 Customizing the EzScan-i Admin User Account

To customize the **EzScan-I** Admin User Account:


- STEP 1.** Click the **EzScan-i** Admin User Account icon : This will open the User Start Screen.
- STEP 2.** Click the **System** icon located in the top right-hand corner of the User Start Screen, to access the **Settings** interface.

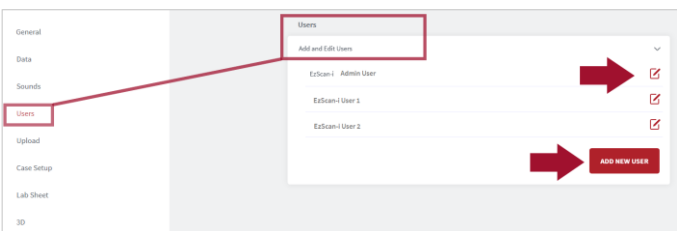

- STEP 3.** Select **Users** in the left-hand **Settings** menu, and click on **Add and Edit Users**.
- STEP 4.** Select the default “EzScan-i” **User profile**, and click on the **Edit** icon to customize the default Admin User account.

**Note:** The default admin ID “EzScan” cannot be modified.

- STEP 5.** In the User Image field, you can click to open the Picture dialog box, to add or take a User Photo.
- STEP 6.** To apply changes, restart the application.

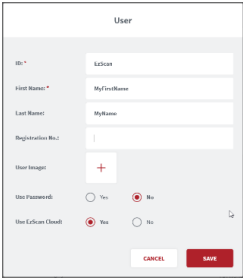

## 4.4 Adding a New User Account

You can use the **EzScan-i** Admin User profile (E) to add new User Accounts.

To add a new User account:

**STEP 1.** In Add and Edit Users, click **ADD NEW USER**:

This opens the **New User** dialog box.

**STEP 2.** Enter User information: email address, first name, last name, registration no. (optional) and photo (optional).


### Password Protecting a User Account

To password protect a User profile (recommended):

**STEP 3.** Select **Yes** in the **Use Password** field.

**STEP 4.** Enter and confirm the password to apply to this user account.

**STEP 5.** To apply **User** changes, close and restart the application.

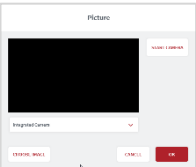


**New User**

Email Address \*

First Name \*

Last Name \*

Registration No.

User Image: 

Use Password: ☐ Yes ☒ No

Use EzScan Cloud: ☒ Yes ☐ No

**New Password**

Password:

Confirm Password:



## 4.5 Accessing Your EzScan-i User Account

User accounts are displayed in the left-hand menu of the EzScan-i Home Page.

To access your User account **Start Screen**:

**STEP 1.** Click on your User profile in the left-hand menu of the EzScan-i Home page.

The **EzScan-i Login** window will open automatically, inviting you to **Login** or to **Create a New Account**.

**STEP 2.** Enter the password and click **SIGN IN**.



## 4.6 Linking a User Account to the EzScanCloud

You can link your **EzScan-i** User account(s) to the **EzScan** solution's dedicated **EzScanCloud** platform. The **EzScanCloud** enables users to easily and efficiently setup and manage file transfers and connections with labs.

**Note:** The **EzScanCloud** platform is designed to facilitate file sharing and manage connections with laboratories. It is **not** a cloud storage service.

To link the **EzScan-i** User profile to an EzScanCloud account:

**STEP 1.** Using the Admin account, In Add and Edit Users, click **ADD NEW USER** to create a User account.

**STEP 2.** Enter account details, and in the **Use EzScanCloud** field, Select **Yes**.

**STEP 3.** Click **Create**, to add the new User account.

**STEP 4.** Click **OK**.


The **EzScanCloud Login** window will open automatically, inviting you to **Login** or to **Create a New EzScanCloud Account**.

A screenshot of the "EzScan Cloud" "Create New Account" form. The form has a title "Create New Account" and a subtitle "Enter your details below and click 'Create Account'". Below the subtitle are several input fields: "user.name@ezscancloud.com" (with a blue bar below it), "Password\*", "First Name\*", "Last Name\*", "Institution Type\*" (a dropdown menu), "Institution Name", "Country\*" (a dropdown menu), "Phone", and "Address". At the bottom left of the form is a red circle with a white plus sign. To its right is the text "Upload an image so your connections can easily recognize your cases (Optional)". At the very bottom of the form is a red button with the text "CREATE ACCOUNT".

# 5. Configuring EzScan-i Settings


## Accessing the Settings Menu

System settings can be accessed by clicking the System icon located in the top right-hand corner of the EzScan-i User Start Screen.

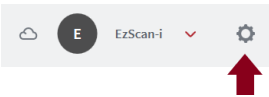

## Settings Groups

To access a specific Settings group, you can select from the left-hand **Settings Menu**, filter by search or scroll down.

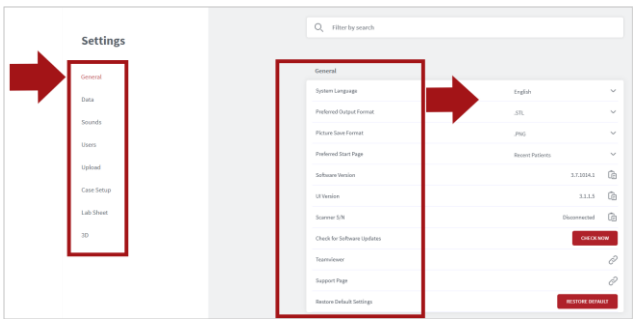


**Note:** System settings are automatically applied to **all users**.

## 5.1 General Settings

The **General** settings menu enables you to select the following:

- **User preferences:** language, default file and image formats etc.
- **Versioning information:** software, UI and Scanner, updates
- **Access support:** access the EzScan support portal

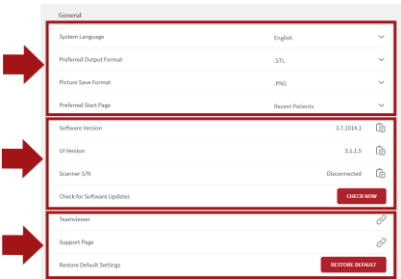

If you make a support request, you will be asked to specify the software version, User Interface (UI) version and Scanner Serial Number (S/N).

These can be copied by clicking on the corresponding copy icon.



## General Settings

Setting	Description
System Language	Select graphical user interface language
Preferred Output Format	Select default 3D file format: .STL, .PLY, .OBJ
Picture Save Format	Select default image format: .PNG, .JPG
Preferred Start Page	Select Recent Patients or Recent Cases default start page
Software Version	Click to copy scanning module software version
UI Version	Click to copy user interface version
Scanner S/N	<p>Click to copy scanner serial number.</p> <p>In accordance with Medical device Regulation (EU) 2017/745, the UDI is composed of a device identifier (UDI-DI) and a production identifier (<b>UDI-PI</b>). The last 6 digits represent the scanner's Serial number (S/N)</p> <p>E.g.: (01)00864206000406(11)191212(10)FR7100AB(21)<b>104246</b></p>
Check for Software Updates	Click <b>Check Now</b> button to manually check for software updates. An automatic check is done at launch
Teamviewer	Click to allow secure remote control of your workstation by a Vatech support technician
Support Page	Vatech online support
Restore Default Settings	Click to restore default settings for the <b>EzScan-I</b> software.

# 5.2 Data Settings

Data settings enable users to:

- Configure Case/Database Import/Export settings
- Carry out Case/Database Import/Export
- Share/Restrict access to the Database
- Program automatic deletion of raw scan data (Auto Clean)

Data

Case/Database Export

EXPORT

Case/Database Import

IMPORT

Include Patient Name in Export Path

☐

Case Export Path

▼

Activate Automatic Export of Data

☐

Exocad DentalCADApp Path

▼

Share Database With All Users

☐

Auto Clean Raw Scan Data

☐

Auto Clean Raw Scan Data older than

1 month

▼

## Case/Database Export

To configure **Case/Database Export** settings:

- STEP 1.

Click on the **EXPORT** button to select the **Export folder** linked to your **EzScan-i Data** folder.

Select Cases

Case Date

Patient ID

Name

Date of Birth

Gender

Case ID

1/4/2022 12:55 PM	0000000	Scan	10/4/2021	Not Specified	187
1/5/2022 3:54 PM	0000000	Scan	10/4/2021	Not Specified	188
1/5/2022 3:54 PM	0000000	Scan	10/4/2021	Not Specified	189
12/30/2021 4:34 PM	0000000	Scan	10/4/2021	Not Specified	184
12/30/2021 18:35 AM	0000000	Scan	10/4/2021	Not Specified	183
10/27/2021 12:41 PM	0000000	Scan	10/4/2021	Not Specified	182
10/25/2021 2:40 PM	0000049	Scan	10/4/2021	Not Specified	181
10/25/2021 12:28 PM	0000049	Scan	10/4/2021	Not Specified	180

CLOSE

OK

Case/Database Export

Case/Database Export

Selections

All Cases

Selected Cases

CLOSE

EXPORT

Case/Database Export

EzScan-i Data folder

C:\ProgramData\EzScan-i

Export folder:

C:\Program Files\EzScan-i\Export

Selections

All Cases

Selected Cases

CLOSE

EXPORT

To export **All** or **Selected Cases** to the Export folder:


- STEP 2.

Click **Selected Cases** and **EXPORT** to access the **Select Cases** window.

- Use **[CTRL]+left-click** to select multiple cases.
- Use **[CTRL]SHIFT+left-click** to select a range of cases.

All files related to the selected case(s) will be exported to the folder configured in Data Settings.

**Note:** The export destination folder must be empty.

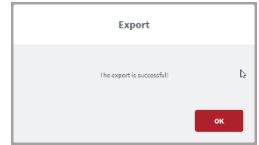

## Activate Automatic Export of Data


When this feature is activated in **Data Settings**, case data is automatically exported to **EzDent-i** without having to click **Export** in the **Case Finalization** page.

## Case/Database Import

In **Data Settings**:


**STEP 1.** Click on the **IMPORT** button to select the Import Folder linked to your EzScan-i Data folder.

You can import **All** or **Selected Cases**.

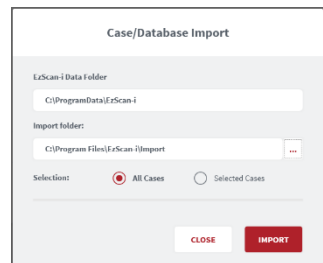

## Exocad DentalCADApp Path

In **Data Settings**, click on **Exocad DentalCADApp Path** to select the export path to your Exocad CAD application.

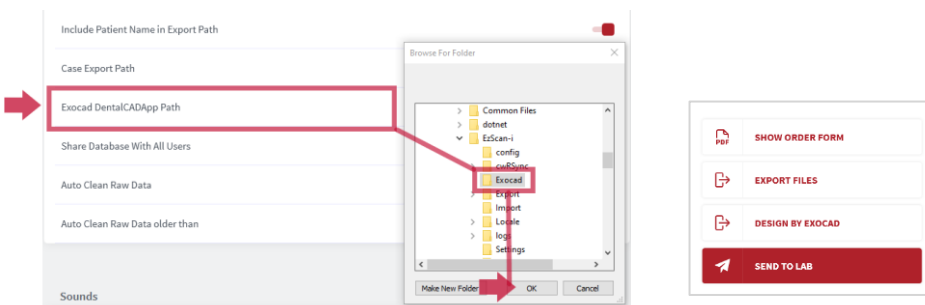

**Note:** When the **Exocad DentalCADApp** export path is configured in **Data Settings**, the **Design BY Exocad** export option is displayed in the **Case Finalization** page.

See below: [section 0](#)



## Finalizing A Patient Case

# Auto Clean Raw Scan Data

**Auto Clean** settings enable you to automatically clean project data for cases after 1 week, 1 month or 3 months.



Auto Clean will delete raw scan data, while still keeping case data sets available for review (indication and prescription remain available for cleaned cases).


Data Setting	Description
Case/Database Export	Click on the <b>EXPORT</b> button to select the <b>EzScan-i Data folder</b> and <b>Export</b> folder, and to export <b>All</b> or <b>Selected Cases</b>
Case/Database Import	Click on the <b>IMPORT</b> button to select the <b>EzScan-i Data folder</b> and <b>Import</b> folder, and to import <b>All</b> or <b>Selected Cases</b>
Include Patient Name in Export Path	Select this option to include patient name in the export path. Deactivated by default
Case Export Path	Click to specify the default Case Export folder
Activate Automatic Export of Data	When selected case files will be exported automatically to the <b>EzDent-i</b> application.
Exocad DentalCADApp Path	Click to define <b>Exocad export file path</b> . <b>NOTE:</b> The <b>Design By Exocad</b> option is displayed in the <b>Case Finalization Export</b> options only when the export path is configured
Share Database With All Users	Activate this feature to share or restrict Database access
Auto Clean Raw Scan Data	Activate this feature to delete scan data automatically after a predefined period. When you activate <b>Auto Clean Raw Scan Data</b> , data for deleted scans is only available in review mode. Case data can be reviewed and shared, but it is no longer possible to update the scan
Auto Clean Raw Data older than...	Select predefined period after which scan data will be deleted automatically: 1 week, 1 month, 3 months. The <b>default period</b> is <b>3 months</b>



# 5.3 Sound Settings

Sound settings enable you to activate/deactivate the Sound Guidance feature during scanning, and to preselect sound effects and volume.

Note: if the computer's volume is deactivated or muted the user will not hear the sounds.

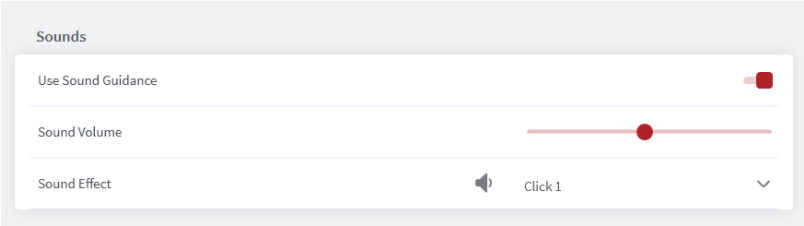

## Audio-guidance

The following audio-guidance sounds are available when scanning:

- One sound for upper arch when registered
- One sound for lower arch when registered
- One sound for both and with sufficient data ( just before/after user stops scanning )


## Sound Settings Description

Sound Setting	Description
Use Sound Guidance	Activate to use Sound Guidance during scanning (Recommended)
Sound Volume	Preselect volume
Sound Effect	Preselect sound effect

# 5.4 User Settings

In User settings you can Add or Edit Users using the administrator account.

User dossiers can be password protected, and linked to an EzScanCloud account.

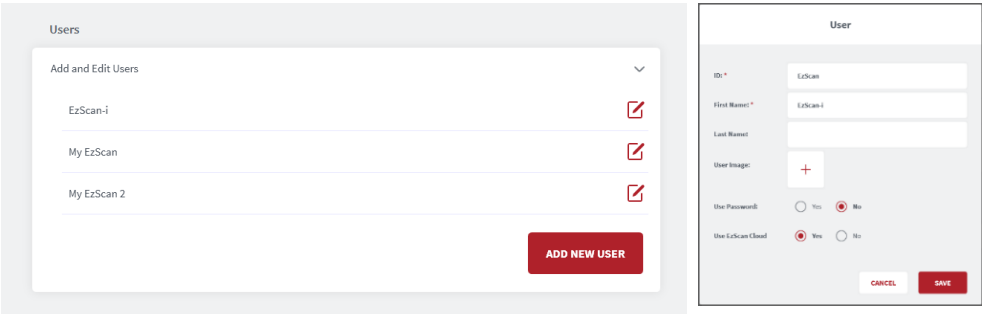

## Add And Edit Users


- Click the right-hand Edit icon to enter/modify User information: email, name and photo.
- Select Use Password to password protect the User account.
- Select Use EzScanCloud to link to an EzScanCloud account.

## 5.5 See above: section 4.6 Linking a User Account to the EzScanCloud

# 5.6 Upload Settings

The Upload setting enables users to select a default lab to transfer files to in their EzScanCloud account.

The default connection will be selected automatically when preparing orders to send to a lab. See below:

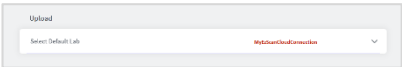

Finalizing A Patient Case.

## 5.7 Case Setup Settings

**Case Setup** settings enable you to adapt the **Case Setup** page to your ordering, restoration and patient requirements.

The screenshot shows the 'Case Setup' configuration page. It includes settings for 'Teeth Numbering System' (set to 'FDI World Dental Federation'), 'Shade System' (set to 'Vita Classic'), and two toggle switches for 'Hide Patients Name' and 'Hide Patient Name When Sending To The Lab', both of which are currently turned off. A red rectangular box highlights the 'Restoration Selector' section, which is set to 'Standard'. Below this, there are two rows: 'Edit Restorations' and 'Edit Implants', each with a red 'EDIT' button. A large red arrow points from the left towards the 'Restoration Selector' box, and two smaller red arrows point from the right towards the 'EDIT' buttons.

### Select Teeth Numbering System

In **Teeth Numbering System**, click to select the notation system of your choice:

- FDI World Dental Federation
- Universal Numbering System
- Palmer Notation Method
- Palmer Notation (digital)
- Primary Tooth Numbering System (for pediatric restorations)

### Select Shade System

In Select Shade System, click to select the desired Shade System: **Vita Classic** or **Vita 3D-Master**.

The screenshot shows a dropdown menu for 'Shade System'. The menu is open, showing two options: 'Vita Classic' and 'Vita 3D-Master'. 'Vita Classic' is selected, indicated by a red checkmark next to it. A red rectangular box highlights the selection area.

### Hide Patient Name

Activate the **Hide Patient Name** feature to anonymize patient identity. This can be useful to protect patient privacy or to use anonymous case examples for training and presentation purposes.


### Hide Patient Name When Sending to the Lab

Enables you to configure settings so Patient Names are anonymized by default when sending patient cases to Labs. Click slider to the *right* to

*activate , to the left to deactivate .*

## Restoration Selector

The **Restoration Selector** setting allows you to select **Standard** or **Alternative** organization of restoration types, enabling you to adapt the management of available restoration options to your specific restorative needs.

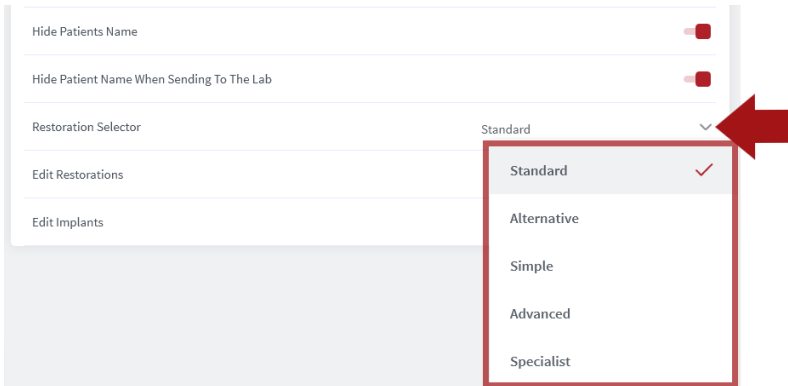

The screenshot shows a settings panel with several toggle switches and a dropdown menu. The 'Restoration Selector' dropdown is open, displaying a list of options. A red arrow points to the dropdown arrow icon.

Setting	Status
Hide Patients Name	On
Hide Patient Name When Sending To The Lab	On
Restoration Selector	Standard
Edit Restorations	On
Edit Implants	On

Standard

Alternative

Simple

Advanced

Specialist

In **Settings**, select **Case Setup**, and in the **Restoration Selector** field, click to select an option:


- **Standard**
- **Alternative**
- **Simple**
- **Advanced**
- **Specialist**

When you select a **Restoration Selector** in **Case Setup** Settings, the **Restoration Editor** window will automatically reflect this choice.

**Note:** *When you select restorations using a selector, you cannot switch to another selector in the same patient case without first deleting selected restorations.*


### Restoration Selector: Alternative

When the **Alternative** option is selected in **Settings**, in the **Case Setup** page the **Choose Restoration Type** selector is displayed as shown:

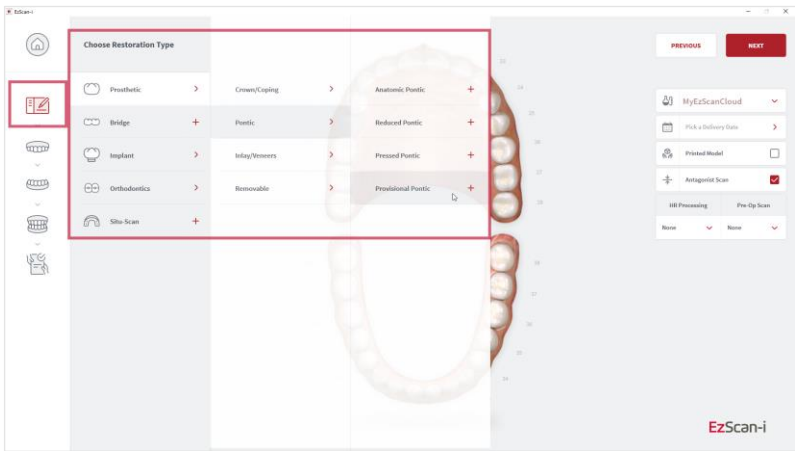

### Restoration Selector: Standard


When the **Standard** option is selected in **Settings**, in the **Case Setup** page the **Choose Restoration Type** selector is displayed as shown:

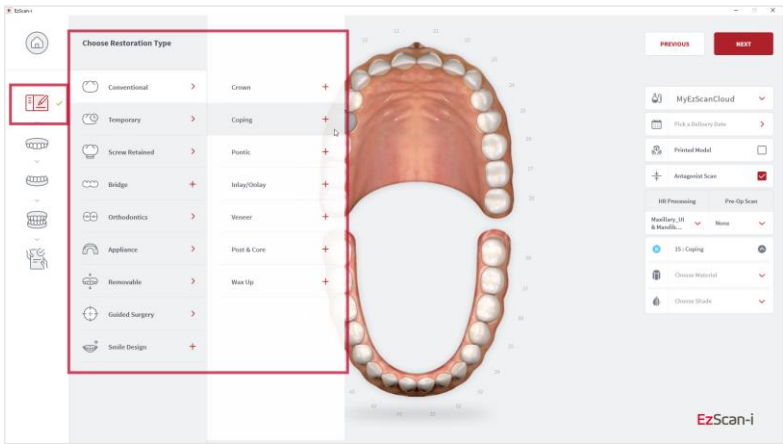

### Restoration Selector: Simple


If the main work of your practice is limited to crown, coping or inlay/onlay preparations, a **Simple** option is proposed.

When the **Simple** option is selected in **Settings**, in the **Case Setup** page the **Choose Restoration Type** selector is displayed as shown.

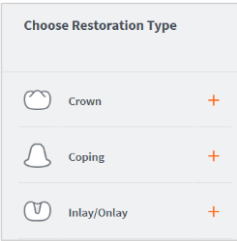


**Restoration Selector: Advanced/Specialist ¶**

When the **Advanced** Restoration Selector option is selected in **Settings**, in the **Case Setup** page the **Choose Restoration Type** selector is displayed as shown:

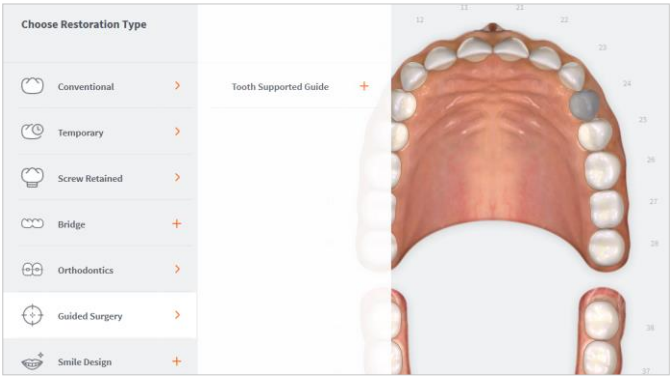

When the **Specialist** Restoration selector option is selected in **Settings**, in the **Case Setup** page the **Choose Restoration Type** selector is displayed as shown:

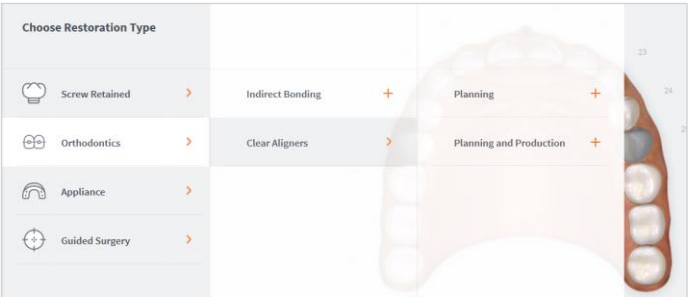


## Edit Restorations

The **Restoration Editor** enables you to **select/deselect** the **Restoration types and materials** available when filling out Case Setup ordering details\*.

To **Edit Restorations** options:

- STEP 1.** Click Edit to open the Restoration Editor.
- STEP 2.** Click to Select/Deselect available Restorations and/or Materials options\*

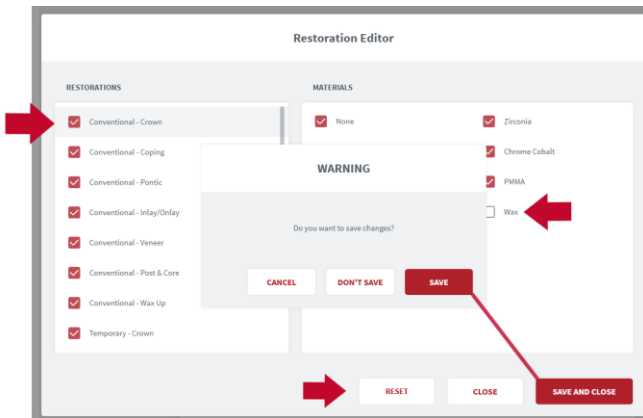

- STEP 3.** Click **Reset** to reset to default user settings.
- STEP 4.** Click **Save and Close** to save changes.  
Click **Close** to close without saving changes.
- STEP 5.** **Restart** the EzScan-i application to apply changes.

**\*Note:** Please verify with your dental lab or service provider about capabilities to produce particular indications.




# Implants Editor

In **Case Setup** settings/**Edit Implants**:

**STEP 1.** Click Edit to open the Implants Editor.

The **Implants Editor** enables users to customize the Manufacturers, Systems and Connection types available when filling out order forms.

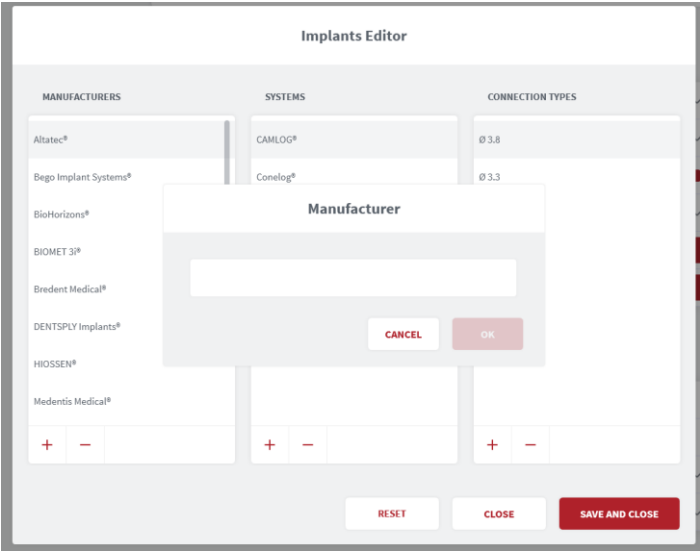

## Add To/Delete From Implants Library

Elements may be added to or deleted from the Implants library, or their order modified, by clicking on the icons at the base of each column.

**STEP 2.** Click **Reset** to reset to default user settings.



**STEP 3.** Click **Close** to close without saving changes.  
Click **Save and Close** to save changes.

**STEP 4.** **Restart** the EzScan-i to apply changes.

## Case Setup Settings Description

Case Setup Setting	Description
Teeth Numbering System	Select order form teeth numbering system
Shade System	Select order form Shade System
Hide Patient's Name	Activate this feature to anonymize patient identity. This can be useful to protect patient privacy and to use anonymous case examples for training and presentation purposes.
Hide Patient's Name When sending to the Lab	Enables you to configure settings so Patient Names are anonymized by default when sending patient cases to Labs. Click slider to the right to activate , to the left to deactivate .
Restoration Selector	<p>Enables you to choose from two different Restoration selectors, Standard and Alternative.</p> <p>Note: When you select restorations using a selector, you cannot switch to the <b>other</b> selector in the <b>same patient case</b> without first deleting selected restorations.</p>
Edit Restorations	The Restoration Editor enables users to customize Restoration types and materials available when filling out Case order forms.
Edit Implants	The Implants Editor enables users to Customize the Manufacturers, Systems and Connection types that are available when filling out order forms.

# 5.8 Lab Sheet Settings

Lab Sheet settings enable users to customize Order forms for use by their organization, by adding the name, address and corporate logo to their Lab Sheets, and selecting a page format (A4/US Letter).

Lab Sheet

Institution Name

Institution Address

Page Size

A4

Institution Logo

# 5.9 3D Settings

3D Settings enable you to select live scan and 3D projection options.

3D

Hole Filling

Hole Highlighting While Scanning

3D Projection

Perspective

Enable Colored ROI Overlay During Bite Alignment

Enable High-Resolution Finalization

Enable Scan Optimization While Scanning

Adapt Live View

Default

## Hole Filling

When this feature is activated the 3D scanning software automatically fills any regions in the scan or “holes” where data is missing. Holes are filled when the user stops scanning and the model is displayed on the screen.

## Hole Highlighting While Scanning

When automatic hole highlighting is activated, this option highlights those areas during the live scan. This enables the user to see where holes are and complete them.

### 3D Projection

Click to select either Perspective or Parallel default view of scanned arches.

**Note:** This option is available in the 3D viewer after Finalization, and in the Case Preview page. It is not available while scanning.

### Enable Colored ROI Overlay During Bite Alignment

Select to display colored ROI overlay during Bite Alignment. Activated by default. Triggers a green/red overlay during scan phase to indicate actively gathering data.

### Enable High-Resolution Finalization

Select to enable default high-resolution finalization. For all indications, It is possible in the **Case Setup** page to modify the setting per case.

### Enable Scan Optimization While Scanning

When activated, carries out scan optimization in the background during the scan.

**Note:** *This setting is particularly recommended for new users, or for users who encounter difficulties during scanning.*

### Adapt Live View


Adapt Live View window to view **Scan From Front** or view **Scan From Behind**, to adjust the orientation of the live view based on the position adopted by the doctor.

## 3D Settings Description

3D Scan Setting	Description
Hole Filling	3D scanning software automatically fills any regions in the scan or “holes” where data is missing. Holes are filled when the user stops scanning and the model is displayed on the screen.
Hole Highlighting While Scanning	Highlights those areas where holes are located during the scan. This enables the user to see where holes are and complete them.
3D Projection	Select either Perspective or Parallel default view of scanned arches
Enable Colored ROI Overlay During Bite Alignment	Select to display colored ROI overlay during Bite Alignment. Activated by default. This triggers a green/red overlay during scan phase to indicate actively gathering data.
Enable High-Resolution Finalization	Select to enable default high-resolution finalization
Enable Scan Optimization While Scanning	<p>When activated, carries out scan optimization in the background during the scan.</p> <p>Note: This setting is particularly recommended for new users, or for users who encounter difficulties during scanning.</p>
Adapt Live View	<p>Adapt Live View window to view Scan From Front or view Scan From Behind, to adjust the orientation of the live view based on the position adopted by the doctor.</p> <p>For demonstrations and models it is recommended you select the default option (view Scan From Front).</p>

# 6. Managing Cases in EzScan-i

Patients and their associated case files can be accessed in the EzScan-i homepage. All patient information can be modified in EzDent-i.

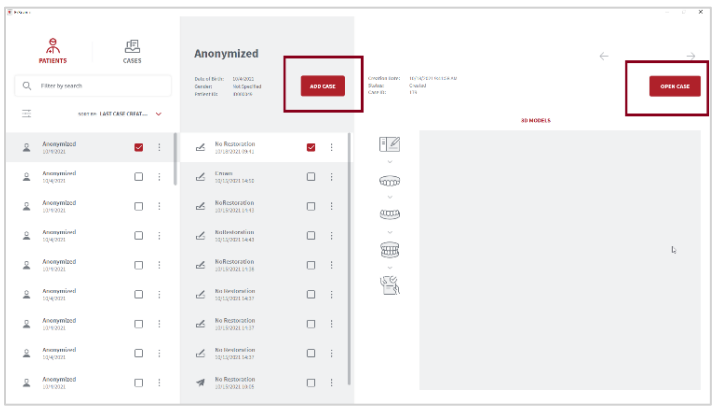


If Patient ID does not already exist in the EzScan database, EzScan-i creates a new Patient entry using Patient ID, names, date of birth and gender provided by EzDent-i.

## 6.1 Managing Patient Profiles

### Filtering Patient Profiles

**STEP 1.** Click on the **PATIENTS** icon in the left-hand menu.

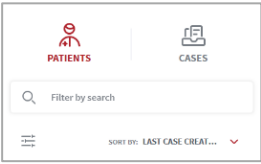


To select the list of Patient profiles:

To organize patient profiles:

**STEP 2.** Click Last Case Created or Name.

To filter patient profiles by **Gender**, **Creation Date** range and/or **Date of Birth**:

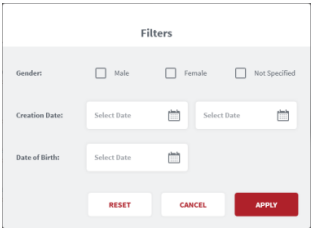

- STEP 3.** Click the **Filter** icon, and in the Filters dialog box select filter details.
- STEP 4.** Click **Apply** to apply the filter, or
- STEP 5.** Click **Cancel** to cancel any modifications made to the filter.

An active filter is indicated by a circle as shown.

To remove an active filter:


- STEP 6.** Click **Reset**.



### Selecting A Patient Profile

- To **Select** a Patient profile:

- STEP 1.** Tick the patient checkbox as shown.

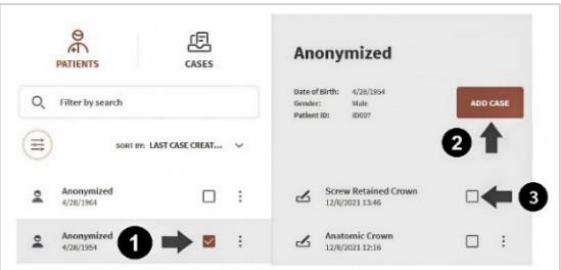

### Deleting Patient Profiles

- To delete a Patient profile:
- STEP 2.** Click on the 3-point icon to the right of the Patient folder, and select **Delete**.

*Note:* Only patient profiles with no cases can be deleted. Cases linked to a patient must be deleted first.

### Anonymizing Patient Data



You can anonymize patient data before sharing patient files with labs.

To anonymize patient data:

- STEP 1.** Go to Settings.

**STEP 2.** In the Settings Menu select **Case Setup** settings.

**STEP 3.** Activate the **Hide Patient Names** option.

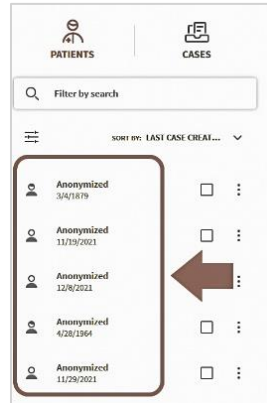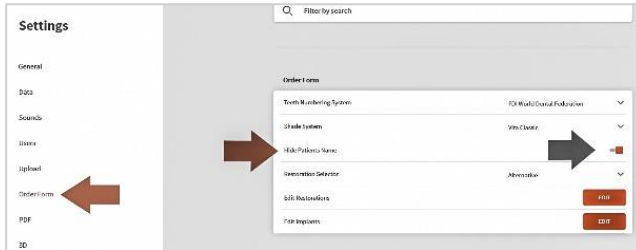

To anonymize patient data when sending an order to a lab:

**STEP 4.** Click to Activate the Hide Patient Name When Sending to the Lab option.

## 6.2 Managing Patient Cases


### Previewing a Patient Case

The **Case Preview** page provides detailed case information at-a-glance: creation date, case ID, restoration types, status, and any 3D models and 2D images associated with the case...

To preview a patient case:

**STEP 1.** Select a patient in the left-hand menu.

**STEP 2.** Select a case in the list of cases associated

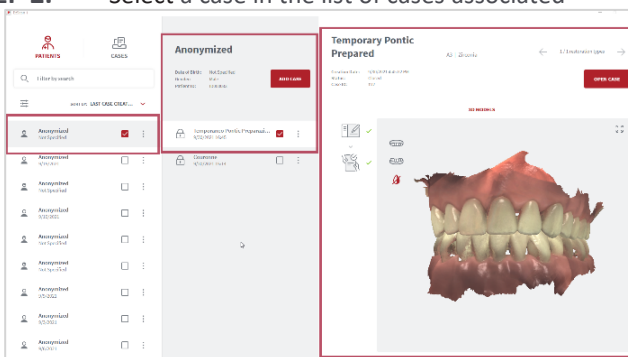



The Case Preview is displayed as shown.

## Opening a Case

To select a patient case,




**STEP 3.** Click Open Case in the Case Preview window:

This will open the **Case Finalization** page.

To directly access any step of the **Case Workflow**:

**STEP 4.** Click on the relevant icon in the left-hand menu.

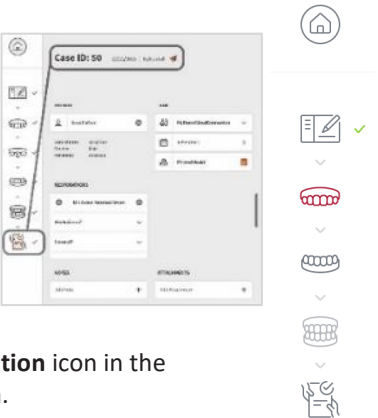

## Case Status

The **Case status** is displayed in the **Case Finalization/Review** page. To access, click on the **Finalization** icon in the lefthand menu, or select a Case in your **User Start Screen**.

## Case Status Icons

Case Status		Description
	Created	Case is created and currently in progress.
	Scanned	Case is scanned and finalized.
	Uploaded	Case is uploaded to lab via EzScanCloud .
	Closed	Case is closed. When the <b>Auto Clean Raw Scan Data</b> option is enabled in <b>Data Settings</b> , cases for which raw scan data is deleted are automatically closed.


## Accessing the Case Setup page

Adding a new case automatically opens the **Case Setup** page, and the icon is highlighted in the EzScan-i Scan Workflow menu.



## 6.3 Setting Up a Case in EzScan-i

The following walkthrough shows the typical steps and options available in the EzScan-i Scan Workflow.

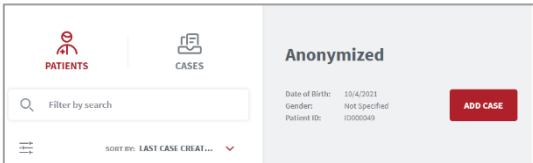

### 1. Add a New Case

To add a new patient case:


**STEP 1.** Select a patient and in the **User Start Screen** and

**STEP 2.** In the **Case Preview** page, click **Add Case**:

This opens the EzScan-i **Case Setup** page.

### 2. Using the Case Setup Page

When you open the **Case Setup** page, the **Case Setup** icon is highlighted in the EzScan-i left-hand Menu.

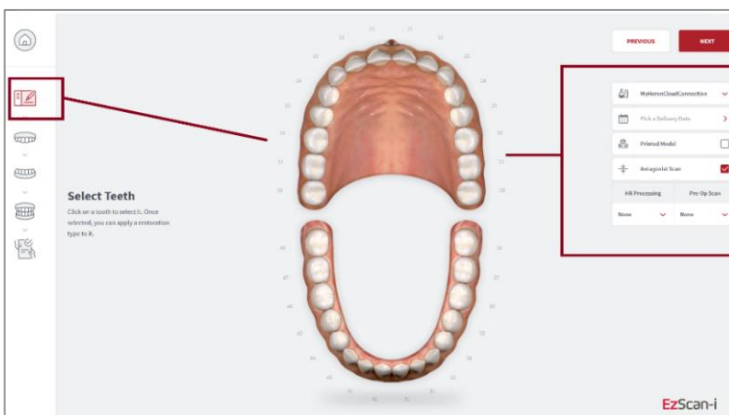

The left-hand **Workflow Menu** will adapt automatically to the Restoration Type(s) selected for the current patient case.

You can select a default lab from those connected to your EzScanCloud account in system **Error! Reference source not found..**

### 3. Select tooth/teeth for restoration

**STEP 3.** Click on the tooth or teeth to select for restoration.

The **Choose Restoration Type** menu displays automatically.

**Selecting a bridge:**

To select teeth to be bridged:


**STEP 4.** Select a tooth and then **Hold [CTL]** to select a second tooth.

Bridged teeth are displayed as shown.



### 4. Choose Restoration Type

**STEP 5.** Click on a **Restoration Type** in the left-hand menu: options available for the selected restoration type are displayed automatically.

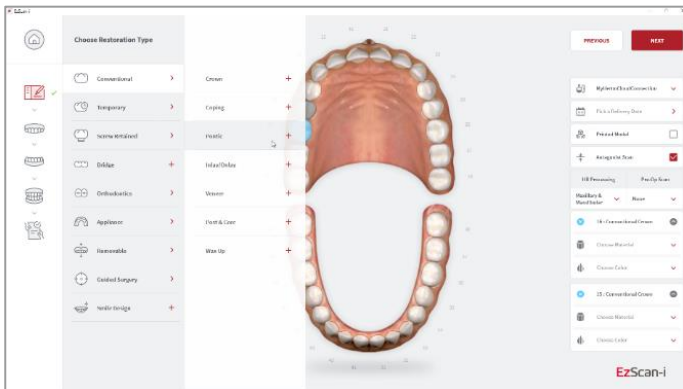

**STEP 6.** Select the Restoration Type: the tooth to which the restoration is applied is highlighted in blue/green.

To change the selected restoration type:

**STEP 7.** Right-click on the selected tooth to delete and then select another restoration type.

Available options for the selected Restoration Types are displayed in the right-hand **Order Form** menu.

### Configuring a default Restoration Selector

You can choose between Standard and Alternative Restoration Selector options in Order Form Settings. See above:

### Hide Patient Name When Sending to the Lab

Enables you to configure settings so Patient Names are anonymized by default when sending patient cases to Labs. Click slider to the *right to activate* , to the *left to deactivate* .

Restoration Selector.


**Note:** When you select restorations using a default selector, you cannot switch to an alternative selector in the **Case Setup** page without first deleting already selected restorations.

## 5. Select Restoration Options

Select Restoration options for the chosen Restoration type in the right-hand **Order Form** menu.

- STEP 1.** Select Manufacturer\*
- STEP 2.** Select System
- STEP 3.** Select Connection Type
- STEP 4.** Select Material
- STEP 5.** Select Shade

Restoration options

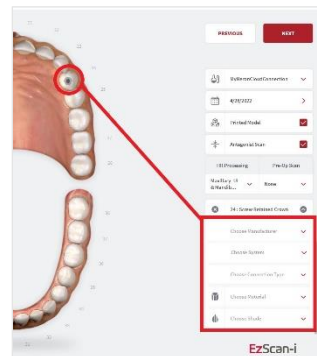

Restoration options		Description
Manufacturer		Available Manufacturers
System		Available Manufacturer Systems/Models
Connection Type		Available implant Connection types and sizes
Material		Available printed model materials
Shade		Available shade systems and shades

You can customize Order Forms to specific ordering, restoration and patient requirements in

Case Setup Settings.

6. Enter Order Form details

Enter Order Form details in the top right-hand menu of the Case Setup page:



Patient ID

A direct scan can be associated with an existing Patient ID by clicking on the Patient ID field and selecting a Patient ID from the drop-down menu.


To create a new Patient ID, click **ADD NEW PATIENT** in the drop-down menu and enter patient details.

LAB

Specify the laboratory to which to send your order.

**Note:** Before selecting a lab, you must first set up a connection with the lab via the **EzScanCloud** platform.

See above: section 4.6 [Linking a User Account to the EzScanCloud](#)

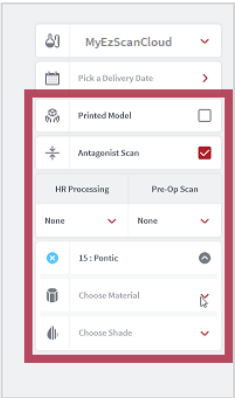

Order Form details	Description
Patient ID	Patient Name or Anonymized ID
EzScanCloud Connection	Connection to lab configured in the EzScanCloud .
Delivery Date	Requested order delivery date
Printed Model	Order a printed model
Antagonist Scan	Include a scan of the opposing teeth (selected by default)
HR Processing	Use High Resolution processing
Pre-Op Scan	Include a pre-operative scan

The antagonist can be de-selected if not required.

You can customize Order Forms to your ordering, restoration and patient requirements. See above: [section 0](#)






Lab Sheet Settings.


# 7. Scanning with EzScan

## 7.1 Before You Scan


Before scanning, take time to familiarize yourself with the Scanning tools and features available in the EzScan solution and the EzScan-i interface.

### Scan Tools Overview







**Scanning Tools**  
**Start Scan**  
Start/pause scan.




**Reset**  
Reset will delete the current scan and associated files.




**Adjust Zoom level**  
You may change the zoom level.




**Center Scan on Screen**  
Moves the scan to the center of the screen for ease of use.




**Color-Based Quality Map**  
Toggles between color/grayscale display and Quality Map.  
The Quality Map feature enables the user to assess in real-time if enough data was collected in the area of the scan.



**Live View Screenshot**  
Take a Live View screenshot during the scan simply by pressing the **C** key or clicking on the **Screenshot** icon.




**Auto-Realignment**  
This feature enables the user to optimize alignment of scans, in preparation for Bite Alignment or for Finalization.




## 7.2 Using the Scan Tip

Each scanner is delivered with three (3) reusable tips, that can be sterilized in an autoclave up to 250 times.

**STEP 1. Attach** the tip with the mirror side facing down-ward. Firmly press the tip onto the scanner until you hear it lock into place.

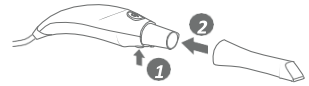


**STEP 2. Detach** the tip by pressing the button located on the bottom of the handpiece (1) while pulling away from the scanner (2).

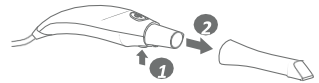

**IMPORTANT:** To avoid scanning errors, make sure the scanner tip clicks securely into place

**STEP 3. Rotate.** To make the scanning process **more comfortable** for maxillary scans, you may remove the tip by pressing the button (1) located on the bottom of the handpiece and rotate the scanner tip 180 degrees (2).

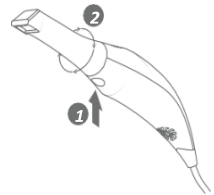

**STEP 4. Start.** To **start** scanning, press the **Start/Stop** button on the top of the scanner, or click the Start/Stop icon in the right-hand **Scan Tools** menu.

For ease of use, hold the EzScan handpiece as shown.



**IMPORTANT:** Make sure the computer on which the HeronClinic™ software is installed is connected to an external power source before starting to scan with the Heron™ IOS.

Do not scan using a battery-powered computer.



## 7.3 An Example Scan Workflow Menu

In the example shown below, based on the selected Restorations, the Scan Workflow indicates 6 Scan Steps and Finalization of the Order.

### STEP 1. Pre-Op Scan

Used to integrate pre-operative scans in the scan workflow.

### STEP 2. Maxillary Scan

To carry out a scan of the Maxillary arch.

### STEP 3. Scan body (Maxillary)

Integrates a scan body step of the maxillary arch in the scan workflow.

### STEP 4. Mandibular Scan

To carry out a scan of the Mandibular arch.

### STEP 5. Scan body (Mandibular)


Integrates a scan body step of the mandibular arch in the scan workflow.

### STEP 6. Bite Alignment

Used to carry out an automatic or manual bite alignment.

### STEP 7. Finalization

Verification and finalization step, before sending the Order to the lab.

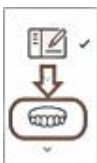


To access the Scan Workflow for your restorations, click **NEXT** in the **Case Setup** page, or click on the next icon in the **Scan Workflow Menu**.

The case workflow will open automatically at the step last worked on.

### Using Arrow Keys to Navigate



For ease-of-use, you can navigate between **Scan Workflow** steps using the **up-down arrow keys** on your keyboard.

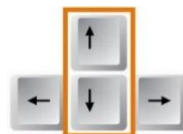

## 7.4 Useful Scanning Tips

### Before You Scan ¶

- **To scan the maxillary arch, rotate tip to face up**

To make the scanning process more comfortable for maxillary scans, remove the tip by pressing the button located under the handpiece and rotate the scanner tip 180 degrees. Make sure the tip locks correctly into place before starting to scan.



- **Avoid moving tissue: retract lips, cheeks & tongue**

- **Take care when using a dental mirror**

Caution is needed when using the back of the mirror for retracting the cheeks as this can get captured in images and lead to slowing down of the scan, as the elements are not native in the scene.



- **Take care when using retractors**

Caution is needed when using different retraction techniques - retractors can cause some issues in anterior area and at the junction of cheek and jaw tissue. Retractors stretch the soft tissue which means on bite step when there is no retraction the soft tissue has changed from the upper and lower scans.



- **Make sure that contact points are captured**



**IMPORTANT:** Before each patient, the scanner's removable tip must be sterilized using an autoclave. See below: Section **Error! Reference source not found.** **Cleaning and Sterilizing Tips.**

## When Scanning...

- **Keep close to teeth**  
(recommended distance from tip to tooth is 0-12 mm)
- **Scan slowly and smoothly**, to avoid interrupting the 3D reconstruction (minimum 1 second per tooth)
- **Keep teeth in center of the field of view**
- **Angle the scanner tip** to take in **as much gingiva as possible**  
(bite alignment is based on gingival data)
- **Avoid extraneous items** (lips, cheeks, tongues, gloves, etc.)
- **Do not hesitate to reset scan if necessary**

## Important Notice

Before launching a scan, make sure that:

- The **EzScan** Scanner is correctly connected to the computer via the USB 3.0 cable provided and the **EzScan-i** software is running.
- The computer on which the **EzScan-i** software is installed is connected to an external power source.
- The computer on which the **EzScan-i** software is installed meets **Minimum Computer Requirements**.
- The **EzScan Scanner** is correctly connected to the computer via the **USB 3.0 cable** and the **EzScan-i** software is running.




**IMPORTANT:** If using a laptop computer, **make sure the power supply is connected to a power outlet** and not running on battery power. Failure to do so will mean that the scanner will not have sufficient power to produce images.

On laptop computers, **battery settings in Windows** should be configured to **high performance mode only**, with **no battery saving option** (Settings/System/Battery).

## 7.5 Accessing the Scan Workflow

To access the Scan Workflow:

- STEP 1.** Click NEXT in the **Case Setup** page, or
- STEP 2.** Select the **Maxillary** arch in the left-hand **Menu** by clicking on the icon, or by using the **Down** key on your keyboard.

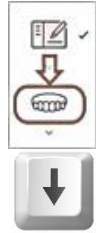


## 7.6 Scanning the Maxillary Arch

- STEP 3.** Start scanning by pressing the **Start/Stop button** on the scanner.
- STEP 4.** Scan the Maxillary arch following the Scanpath Strategy described below:
  - **Section0 -**

- **Recommended Scanpath Strategy Error!**  
**Reference source not found.**

**STEP 5.** Stop scanning by pressing the **Start/Stop button** on the scanner.

The EzScan-i software will process the Maxillary scan data before moving on to the next phase of the Scan Workflow.

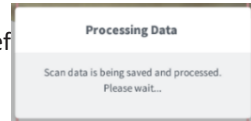

## 7.7 Scanning the Mandibular Arch

If you have selected **Antagonist Scan** in the **Case Setup** page, you will be invited to select the Mandibular arch:

**STEP 6.** Click **NEXT**, or select the **Mandibular** arch in the left-hand **Scan Workflow Menu** by clicking on the icon or by using the **Down** key on your keyboard.



**STEP 7.** Repeat the same scanning strategy as described above for the Maxillary arch...

During the scan workflow, you can use the **Scan Tools** available in the right-hand menu to edit scans. See below:

- **section 7.11- Error! Reference source not found.**



## 7.8 Recommended Scanpath Strategy

### Scanning the Maxillary Arch

When scanning the Maxillary arch, we recommend you scan in the following order:

#### 1. Occlusal – 2. Buccal – 3. Palatal


##### STEP 1. Scan Maxillary Occlusal End-to-End

First scan the **OCCUSAL surface** from molar to molar, with a : **smooth motion**, ensuring **full occlusal surface** is captured for molars and premolars.



This initial path will drive the cross-arch the scan, so **always stay flat on the teeth**.

It may be useful to angle the scanner slightly when you come to the incisor and canine teeth.

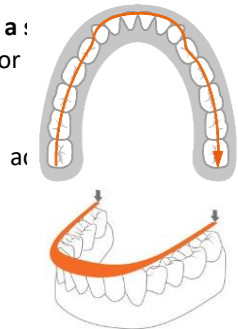


##### STEP 2. Scan Maxillary Buccal LEFT

a. Scan the **BUCCAL area from molar to center line on LEFT side** connection of surfaces:

- Scan with **45°angle** to get part occlusal + part buccal
- Scan with **90°angle** to get last part



b. Scan **gum 3-4mm** in molar/pre-molar area **ON LEFT side**.

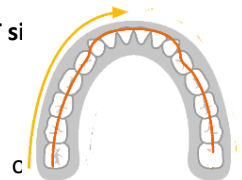


##### STEP 3. Scan Maxillary Buccal RIGHT

c. Scan **BUCCAL area from molar to center line on RIGHT side**, ensuring the connection of surfaces:

- Scan with **45°angle** to get **part occlusal + part buccal**
- Scan with **90°angle** to get remaining part of **buccal**

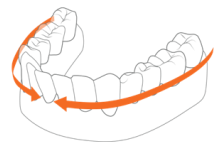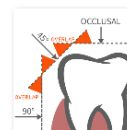

d. Scan **gum 3-4mm** in molar/pre-molar area **ON RIGHT side**



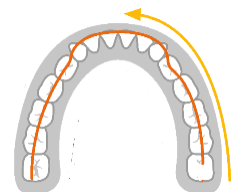


##### STEP 4. Scan Maxillary Palatal End-to-End

Scan the **PALATAL area from molar to molar**, ensuring the connection of

the surfaces (overlap):

- Scan with **45°angle** to get **part occlusal + part palate**

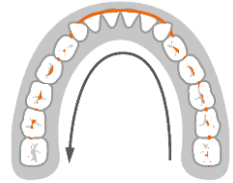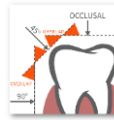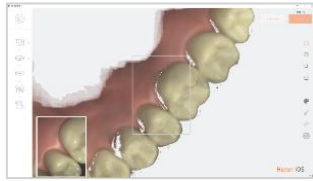

- Scan with **90°angle** to get remaining part of palate

When you have finished scanning:


**STEP 5.** Switch the scanner off using the Stop button

## Processing Data



The EzScan-i software will process the Maxillary scan data before moving on to the next phase of the Scan Workflow.

If you have selected **Antagonist Scan** in the **Case Setup** page, you will be invited to select the Mandibular arch:

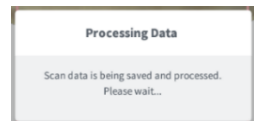

**STEP 6.** Click **NEXT**, or select the **Mandibular** arch in the left-hand **Scan Workflow Menu** by clicking on the icon or by using the **Down** key on your keyboard.



## Scanning the Mandibular Arch


**STEP 7.** Repeat the same scanning strategy as described above for the Mandibular arch.

However, due to the **specific environment of the mandibular arch**, it is recommended you observe the following scan path:

### 1. Occlusal – 2. Lingual – 3. Buccal.

When the scan is completed, the EzScan-i software will process the Mandibular scan data before inviting you to advance to the Bite Alignment phase of the Scan Workflow.

This may take a couple of minutes.

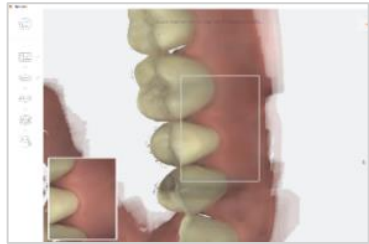

### Next Step




To advance to the Bite Alignment:

- Click **Next**, or
- Click on the **Bite Alignment** icon in the left-hand **Scan Workflow Menu**, or
- Navigate using the **Down** ↓ key on your keyboard.



## 7.9 Scan Memory Use

**Over-scanning** may use up available memory space. When the warning “**Scan memory is running low please conclude scan**” is displayed, no more frames will be added to the scan after this point.

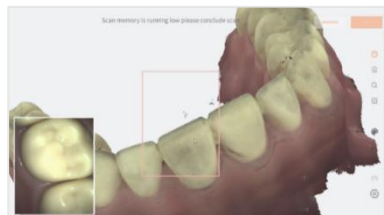


If the result is unsatisfactory, It is recommended you **delete and redo the scan**.



**IMPORTANT:** When you receive the warning “**Scan memory is running low please conclude scan**” no more frames will be added to the scan after this point. If the result is unsatisfactory, it is recommended you **delete and redo the scan**.

# 7.10 Scan Live View

The **Live View** window in the lower left displays what the EzScan-I Scanner is actually seeing. 3D reconstruction of the scan is displayed in the middle of the screen.

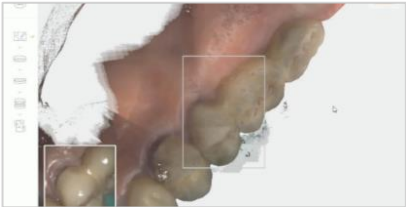

If the scanner loses connection with the reconstructed image, the Live View perimeter frame will be displayed in red.

Return the scanner tip to an already scanned section to resume the scan.

**NOTE:** You can continue to the next scanning area or pause the scanner at any time during the Scanning process without leaving the scanning workflow.

# 7.11 Using Scan Tools

At each step of the scan workflow, you can use the features available in the right-hand **Scan Tools** menu.

Scan Tool	Description
Start Scan	START/STOP/PAUSE the scan.
Reset Scan	Reset will delete the current scan step and associated files.
Adjust Zoom level	Change the ZOOM level of the digital 3D model.
Reset VIEWPORT	Re-center the digital 3D model on the screen.
Quality Map	Toggle between color/grayscale display and <b>Quality Map</b> . The <b>Quality Map</b> feature enables the user to assess in real-time if enough data was collected in the area of interest.

Scan Tool	Description
<b>Live View Screenshot</b>	Take a Live View screenshot during the scan simply by pressing the C key or clicking on the <b>Screenshot</b> icon.
<b>Auto-Realignment</b>	Optimize the alignment of scans, in preparation for Bite Alignment or for Finalization. <b>NOTE:</b> You may also configure the <b>Enable Scan Optimization While Scanning</b> feature in <b>3D Settings</b> .

## 7.12 Using the Quality Map ¶




The **Quality Map** tool enables you to assess in real-time if enough data has been collected in the area of interest of the scan.

To access the **Quality Map** :

1. Click on the *Color Palette* icon in the right-hand tools menu.
2. In the extended menu, click on the *Quality Map* icon.



The digital 3D model will indicate in red those areas where insufficient data has been obtained.

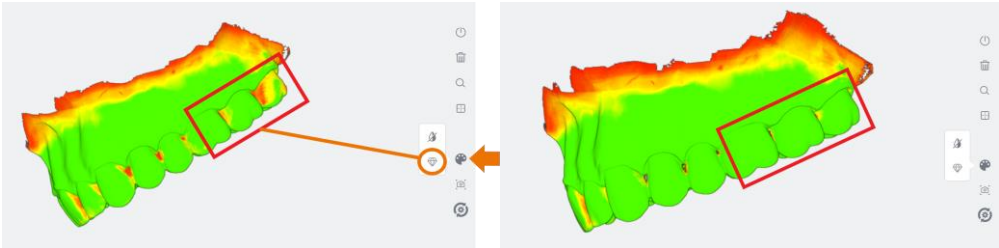

3. Using the **Quality Map** as a guide, rescan the area(s) of interest.

4. To close the **Quality Map** , click again on the *Color Palette* icon.

**NOTE:** The **Color Map** is disabled automatically when you select another Scan Tool or move to the next step in the scan workflow.



**TIP:** Do not over-scan in an effort to recover extra data: it is better to reset and redo the scan if the result is not satisfactory.


## Disable/Enable Captured Color ¶



To make contours more visible, you may wish to disable the natural color of the digital 3D model.

To **Disable Captured Color** :

1. Click on the *Disable Captured Color* icon in the extended *Color Palette* menu.
2. To return to the natural colors of the scan, click again on the *Disable/Enable Captured Color* icon.
3. To close the extended menu, click again on the *Color Palette* icon.

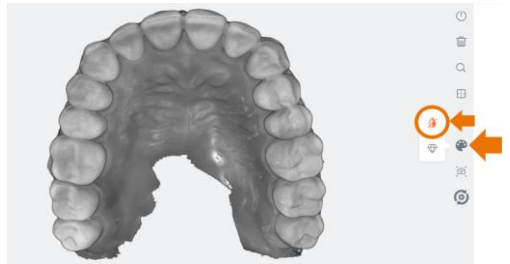

## Taking Live View Screenshots


To obtain Live View Screenshots during the scan:



1. Press the **C** key on your keyboard or click on the icon in the scan tools menu.



2D Live View Screenshots are displayed in thumbnail format in the bottom left-hand corner of the screen.

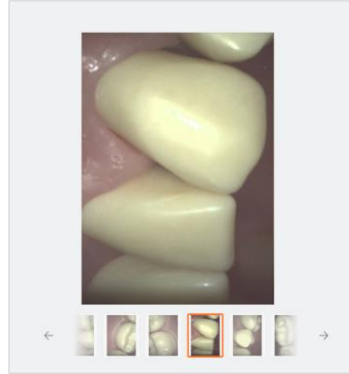

1. Click on a *thumbnail image* to inspect the 2D images in **fullscreen** format.
2. Click on the *Delete* icon to delete as necessary.
3. Click on the *Close* icon in the top right-hand corner to close the fullscreen view.

**Live View Screenshots** taken during the scan can be viewed in the **Finalization** page.

## Using Auto-Realignment ¶

The **Auto-Realignment** tool carries out realignment of the scans, optimizing the scan data based on **Artificial Intelligence (AI)**. To simplify, the scan data is cleaned by discarding frames that are calculated to be misaligned with respect to the totality of the available data.



You are advised to use the **Auto-Realignment** tool **before** proceeding to the **Bite Alignment** step, as optimization may modify the occlusal position of the realigned arches.

### 7.13 Using EzScan Live Scan Tools ¶

During the **Scan Workflow** , **Live Scan Tools** are displayed below the digital 3D Model as shown:



Live Scan Tool	Description
Undercuts Tool	<p>The <b>Undercuts Tool</b> enables you to detect and calculate undercuts:</p> <p><b>User View:</b> enables user to choose the insertion axis that will be used to calculate undercuts on the dental arch</p> <p><b>Auto Detect:</b> determines the optimal insertion axis and displays the undercuts.</p> <p>Undercut areas of the scan model are indicated using a color-coded gradient.</p>
Region Lock	<p>Tool to support the locking of a region while scanning to avoid any further updates of that region.</p>
Cut Tool	<p>Tool enabling you to delete mesh data as required.</p>



Live Scan Tool	Description
Measurement Tool	Tool enabling you to take measurements on the 3D digital model.


# Using the Undercuts Tool



The **EzScan Undercuts Tool** automatically detects and calculates undercuts on the digital 3D model:

- **User View:** enables the practitioner to choose the insertion axis that will be used to calculate undercuts on the dental arch
- **Auto Detect:** determines the optimal insertion axis and displays the undercuts


The insertion axis is indicated by a rod.

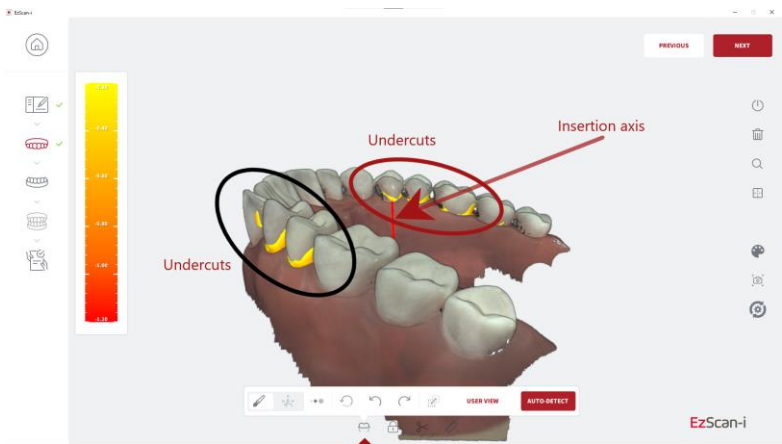

# Undercuts Tool: Expanded Menu ¶


To access the **Undercuts** tool expanded menu:

- Click on the Undercut icon in the **Live Scan Tools** menu, displayed below the 3D digital model.

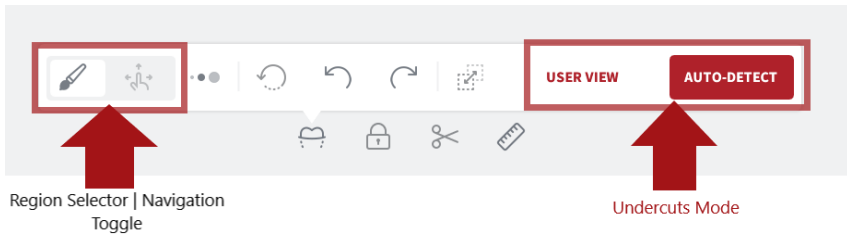

To position the 3D digital model:


- Click on the **Navigation** icon of the **Region selector | Navigation** toggle button.
- Using the **Navigation tool** , *Left-Click+Drag* the 3D digital model to the desired axis of insertion for the restoration.

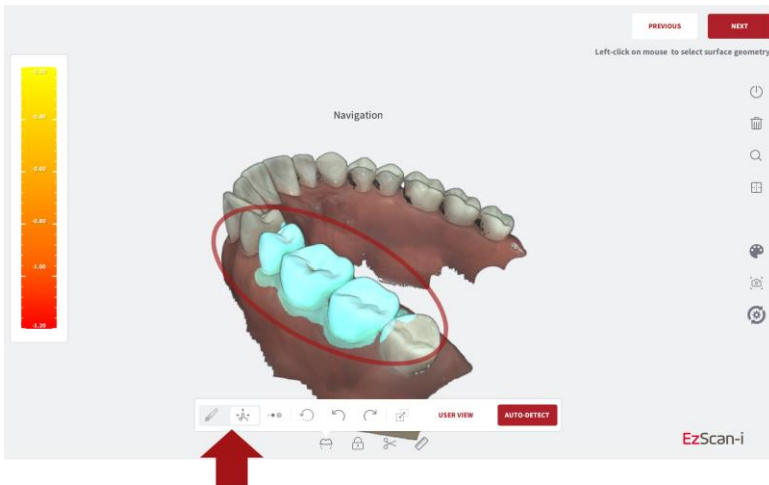

## Selecting a Region ¶

To Select a Region for which to calculate undercuts:

- Click on the **Brush** icon of the **Region selector | Navigation** toggle button.
- Select the **Brush Size** in the expanded **Undercuts** menu.
- Using the **Brush** tool, *Left-Click+Drag* to select the **Region** of interest.

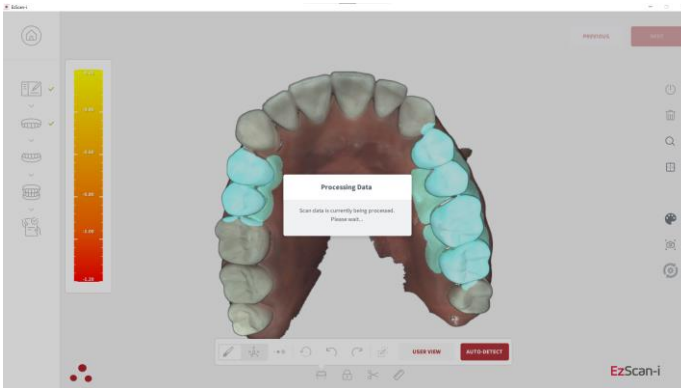

**NOTE:** You can select multiple regions by repeating the same procedure for another region.

## Undercuts: User View ¶

The User View option enables you to choose the insertion axis, used to calculate undercuts on the dental arch.

- Using the **Navigation tool** , **Left-Click+Drag** the 3D digital model to the desired axis of insertion for the restoration.
- Click on the **User View** button.

The Undercuts tool will detect and calculate undercuts based on the chosen insertion axis.

## Undercuts: Auto-Detect ¶

To auto-detect undercuts based on the ideal angle for the arch:



- Click on the **Auto-Detect** button. The Undercut tool will automatically calculate the undercuts for an optimal insertion axis.

Undercut areas of the scan model are indicated using a color-coded gradient.

## Using the Region Lock tool ¶



The **EzScan Region Lock Tool** allows you to lock data for a region/regions while scanning, to avoid any further updates of that region.

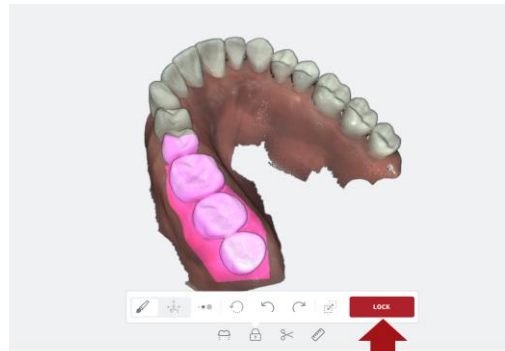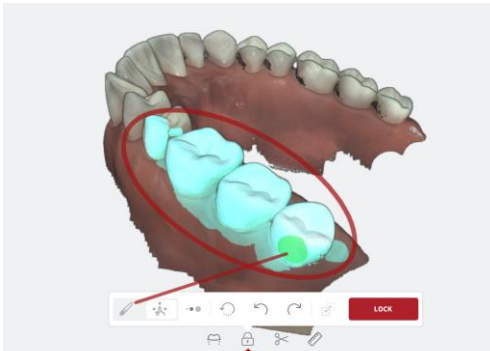


To Select a **Region** to lock:

- Click on the **Brush** icon of the **Region Selector | Navigation** toggle button.
- Select the **Brush Size** in the expanded **Region Lock** menu.
- Using the **Brush** tool, **Left-Click+Drag** to select the **Region** of interest.
- Click on the **LOCK** button.

**NOTE:** You can select multiple regions by repeating the same procedure for another region.

## Using the Trim tool

After scanning, you can use the Trim tool in the Live Scan Tool menu to trim the 3D model and remove any unwanted data.

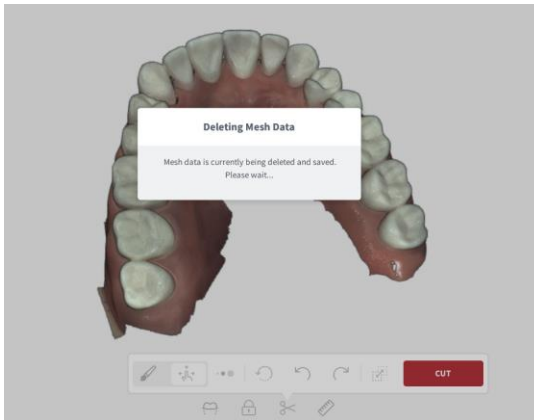

To use the Trim tool:

- STEP 1.** Click on the Trim icon in the **Live Scan Tools** menu.
- STEP 2.** Select the **Trim** brush size.
- STEP 3.** Click on the **Navigation** icon of the **Trim | Navigation** toggle button: using the **Navigation** tool , **Left-Click+Drag** to *position* the 3D digital model.
- STEP 4.** Click on the **Trim** icon of the **Trim | Navigation** toggle button: **Left-Click+Drag** to select the surface area to be removed.
- STEP 5.** Click on the **Cut** button to remove unwanted mesh data.
- STEP 6.** Click **OK** to Confirm Trim Operation.

**IMPORTANT:** Trimmed data cannot be recovered!

## Using the Measurement Tool ¶



- STEP 1.**     **Left-Click+Drag** to position the 3D digital model.
- STEP 2.**     Click on the Measurement icon in the Live Scan Tools menu.
- STEP 3.**     **Left-Click** on a first measurement point.
- STEP 4.**     **Left-Click** on a second measurement point.



The measured distance is displayed in the top right-hand corner of the screen.

## 7.14 Carrying Out A Bite Alignment

The EzScan enables you to carry out **Bite Alignment** based on the previously scanned Maxillary and Mandibular arches. The Bite Alignment can be carried out automatically or manually.

### Accessing Bite Alignment in the Scan Workflow



To advance to the **Bite Alignment** step:

- Click **Next**, or
- Click on the **Bite Alignment** icon in the left-hand **Scan Workflow Menu**, or
- Navigate using the **Down** ↓ key on your keyboard.



### Changing between automatic and manual mode

The Bite Alignment process can be carried out automatically or manually.

You can change between automatic and manual mode at any step of the bite alignment procedure without losing scan data.

### Using audio-guidance

The following audio-guidance sounds are available when scanning the bite segments:

- One sound for upper arch when registered
- One sound for lower arch when registered
- One sound for both and with sufficient data ( just before/after user stops scanning)

Audio-guidance options can be configured in Settings.

For more information see:

- [Section 5.3 - Audio-guidance](#)



**NOTE:** For a better scanning experience, it is recommended to **first position the scanner in the fully open patient mouth** and then ask the patient to bite down.


## 7.15 Carrying out an Automatic Bite Alignment

The Automatic Bite-Alignment feature enables the 3D software to automatically recognize and align the patient's maxillary and mandibular arches based on a brief scan of a segment of the patient's bite in occlusion.

### Recommended Scanpath Strategy

#### Scan Molar/Pre-Molar Segments

For the EzScan-i to automatically recognize the Mandibular and Maxillary arches, it is recommended you:

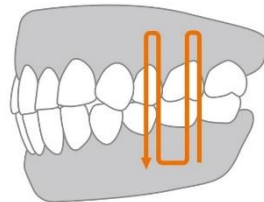

- STEP 1.** Start scanning by pressing the Start/Stop button on the scanner.
- STEP 2.** Scan vertically along a tooth and 1cm of gingiva in the molar/pre-molar region
- STEP 3.** Angle the scanner tip to take in as much gingiva as possible
- STEP 4.** Pause 3-4 seconds on mandibular gingiva
- STEP 5.** Scan up from mandibular gingiva to maxillary gingiva
- STEP 6.** Pause 3-4 seconds on maxillary gingiva
- STEP 7.** Repeat for adjacent tooth as necessary...

**Note:** Take your time when scanning gingival regions. The EzScan software calculates the **3D bite reconstruction** based on **gingival** data.

This makes it possible to successfully scan **edentulous** areas of the arch.





## Auto-locking of bite scan

Once the software recognizes the scanned bite segment:

- A sound signal is heard
- The scan automatically **locks**
- The **initial scanned arches are overlaid**
- A green checkmark displays in the top-left of the screen, indicating you have **finished scanning** that segment.



**STEP 8.** You will be invited to repeat for the opposing side...

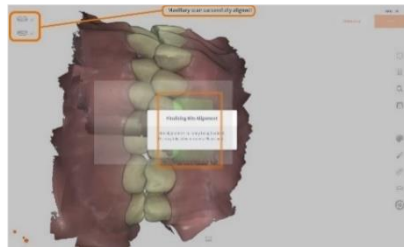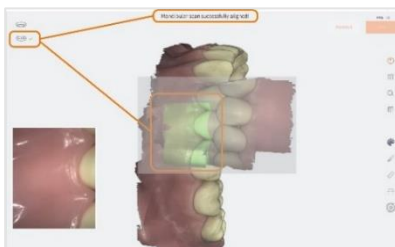

## Bite-alignment using a single bite segment

While it is **recommended**, when carrying out full-arch scans, that you scan **both left and right bite segments**, it is also possible to carry out bite alignment by scanning a single bite segment (left or right).



**STEP 5.** To scan the opposing side, click **OK**; to base the bite alignment on a single bite segment, click **Cancel**.

The **Automatic Bite Alignment** may take a minute or two to process.

**Note:** *In the event of a quadrant scan, you will be asked to scan only the side*



*of the quadrant. When doing full arch scans, you will be prompted to scan bilateral.*

Based on the left and right-side bite scans, the EzScan-i will automatically reconstruct the alignment of the full Maxillary and Mandibular arches in occlusal mode. When **the Automatic Bite Alignment** is completed, the 3D reconstructed image is displayed, and the **Bite Alignment Tools** menu becomes available.

For more information see below: [section 0](#)

**Using Bite Alignment Tools. Error! Reference source not found.**

## 7.16 Carrying out a Manual Bite Alignment

If the **Automatic Bite Alignment** is unsuccessful, due to specific challenges related to the scanned arches that render automatic reconstruction problematic, the **Manual Bite Alignment** tool enables you to **manually** select specific location points on the Maxillary and Mandibular arches.




To Carry out a **Manual Bite Alignment**:

### A. Align Mandibular-Paired Points

**STEP 1.** Click on the **Manual Alignment** icon in the **Bite Alignment tools** menu at the bottom of the screen.

**STEP 2.** Scan a 2-3 teeth and 7-8mm gum segment on the LEFT molar/pre-molar region in Occlusion.

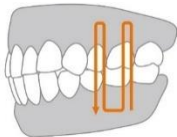


You will be prompted to **select a mandibular paired point** on the segment.

**STEP 3.** Click **Select Point** and choose a point on the **mandibular arch** of the LEFT-side bite scan.

**STEP 4.** When you have positioned the marker, **left-click**: a yellow dot will indicate the selected mandibular-paired point.

**STEP 5.** Click **Confirm Point**.

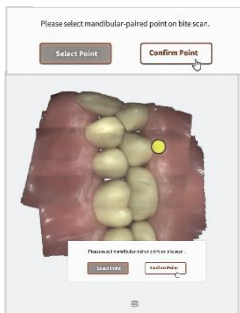

The bite scan segment and the selected mandibular-paired point is displayed automatically in the top right-hand of the screen. You will be prompted to select the corresponding paired point on the mandibular arch.

**STEP 6.** As before, click **Select Point**, left-click: a yellow dot will indicate the selected mandibular-paired point.

**STEP 7.** Click Confirm Point.

You will be prompted to select a maxillary-paired point



When selecting pair points, choose an **easily identifiable surface**, without dense image complexity.

The EzScan-i software calculates the matching position based on **recognizable surrounding surfaces** and not on the **specific point** you have chosen, so you do not need to worry about replicating the **exact position**.

## B. Align Maxillary-Paired Points

**STEP 8.** Click **Select Point** and choose a point on the maxillary arch of the LEFT-side bite scan segment.


**STEP 9.** When you have positioned the marker, left-click: a yellow dot will indicate the selected maxillary-paired point.

**STEP 10.** Click Confirm **Point**.



The bite scan segment and the selected maxillary-paired point is displayed automatically in the top right-hand of the screen.

You will be prompted to select the corresponding paired point on the maxillary arch.

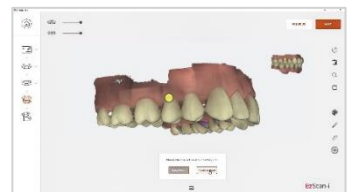

**STEP 11.** Click **Select Point** and choose a point on the maxillary arch of the LEFT side.

**STEP 12.** When you have positioned the marker, **left click**: a yellow dot will indicate the selected maxillary-paired point.

**STEP 13.** Click **Confirm Point** to complete the marker alignment for the LEFT side.

**STEP 14.** You will be invited to start the Bite Alignment based on the chosen paired-points: click **Start**.




### C. Scan a molar/pre-molar bite segment on the opposing side

When you have finished manually aligning one side of the bite, you will be prompted to scan the opposing side.

Carry out the same procedure as described above in **steps 2-14**.

**STEP 15.** Scan a 2-3 teeth and 7-8mm gum segment on the RIGHT molar/pre-molar region in Occlusion...

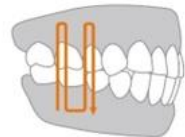

NOTE: While it is **recommended**, when carrying out full-arch scans, to scan **both left and right bite segments**, it is possible to carry out bite alignment by scanning a single bite segment.


See above: **Bite-alignment using a single bite segment**

## 7.17 Auto-Realignment

The **Auto-Realignment** feature enables you to optimize the scans of the maxillary and mandibular arches. The EzScan-i software's AI will recalculate the results using specially developed imaging algorithms, retaining only the most accurate frames for 3D image reconstruction.

**STEP 1.** In the **Tools** menu, click on the **Auto-Realignment** icon. You will be prompted to confirm optimization.

**STEP 2.** Click **OK** to start **Auto-Realignment** optimization of the scans.

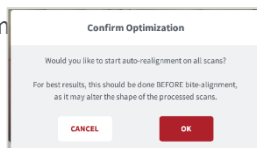

**NOTE:** **Auto-Realignment** may take a couple of minutes.

## 7.18 Using Bite Alignment Tools

When you have successfully completed the Bite Alignment for your patient, you can use the **Bite-Alignment tools** to visually inspect the quality of the alignment scans.

The Bite-Alignment Tool menu is displayed below the 3D reconstructed image of the bite alignment.



### Occlusion Distance Map

To visually check the clearance distance between the maxillary and mandibular arch scans:



**STEP 1.** Click on the Occlusion Distance Map icon in the Bite Alignment Tool menu.



**STEP 2.** Adjust the color map by dragging the visual slider to the desired value.

Contact points are indicated as shown.



### **Open Jaws**

You can use this tool to open the jaws after the bite alignment, to better inspect your scans.



### **Swap Jaws**

Allows the user to swap jaw scans (in case lower was scanned instead of upper or vice versa).




# 8. Finalizing A Patient Case

## 8.1 Finalizing Your Order

When you have finished scanning, you can move to the finalization step of your workflow



**STEP 1.** Click on the **Finalization** icon in the Scan Workflow menu or click **Next**:

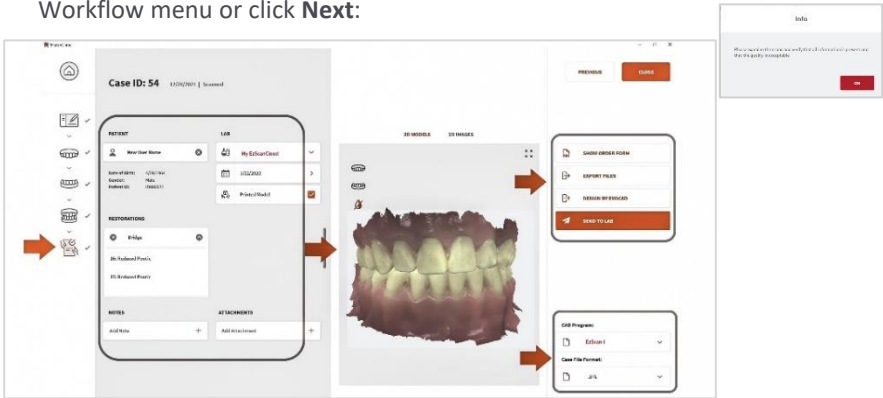

This will launch the **Finalization** process, and then open the **Case Finalization** page.

The **Case Finalization** page enables you to:

1. Review/modify Case Setup Options.
2. Complete **Error! Reference source not found.** (see *below* for details).
3. Edit Case Files using the Case Finalization Tools
4. Send your Order to a Lab , or Export Case Files for integration in a third party system.

## 8.2 Case Finalization Options

The following **Case Finalization Options** are available:

No.	Finalization Option	Description
1	Patient Details	Patient Name, Date of Birth, Gender, Patient ID.

No.	Finalization Option	Description
		<p>For Patient Details options go to: <b>Error! Reference source not found.</b></p>
	Lab Details	<p>Lab connection, Requested Delivery Date, Printed Model.</p> <p>For details of Lab options go to: <b>Error! Reference source not found.</b></p>
2	Order Form & Export	<p>Order Form PDF, Local Export configuration, Send to Lab.</p> <p>For more details go to: <b>Error! Reference source not found.</b></p>
3	Restorations	<p>For details of available Restorations options, go to: <b>Error! Reference source not found.</b></p>
4	3D Models & 2D Images	<p>Preview of <b>3D Model</b> and <b>Live View Screenshots</b> , providing access to <b>Case Finalization Tools</b> :</p> <ul style="list-style-type: none"> <li>- For information on Live View Screenshots taken during the scanning process, go to: <b>Taking Live View Screenshots</b></li> <li>- For information on <b>Case Finalization Tools</b> , go to: <b>Using Case Finalization Tools</b></li> </ul>
5	File Export & Format	<p>CAD Program and Case File Format options: CAD Program Options Case File Format Options</p>



No.	Finalization Option	Description
6	Notes & Attachments	Add relevant notes and files


### 8.3 Using Case Finalization Tools

#### Accessing Case Finalization Tools

Case Finalization Tools are available in the **Case Finalization 3D Models** tab.





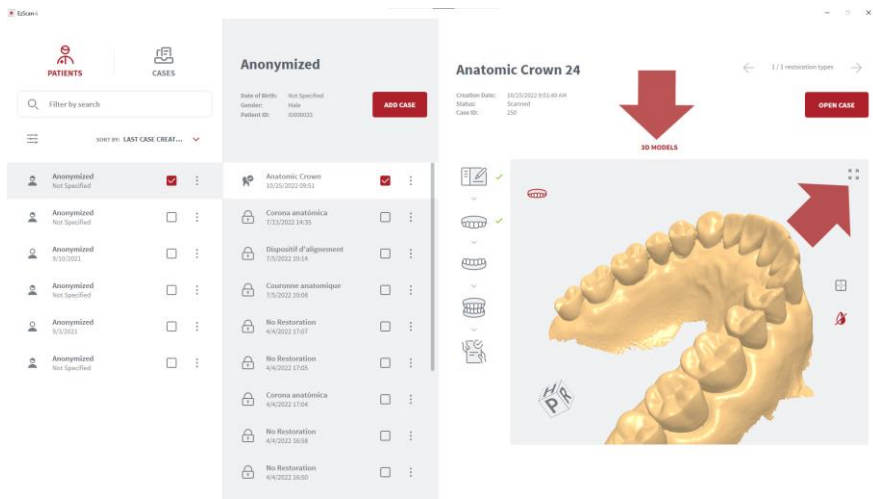



To access **3D Model Case Finalization Tools** :

- STEP 1.** Click on the **3D Models** tab.
- STEP 2.** Expand full-screen to access all view and edit options.



#### Case Finalization Tools

The following **Case Finalization Tools** are available in **EzScan-i**:

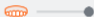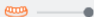
Icon	Case Finalization Tool	Description
	Reset Viewport	Re-center the digital 3D model on the screen.
	Choose Color Display	Choose between natural captured color, grayscale and bi-color display of digital 3D Model.
	Measurement	Tool enabling you to take measurements on the digital 3D Model.
	Margin Line Tool	The Margin Line Tracing tool enables you to trace margin lines on 3D models, in order to enhance the accuracy of lab restoration work.
	Screenshot	Take screenshots of the 3D model as you edit it.
	Model Close Tool	Enables you to prepare the digital model, in order to create a 3D printed model.
	Display Maxillary Arch	Click on the icon to hide/display the Maxillary arch. use the slider to increase/decrease the opacity of the display.
	Display Mandibular Arch	Click on the icon to hide/display the Mandibular arch. use the slider to increase/decrease the opacity of the

Icon	Case Finalization Tool	Description
		display.

## 8.4 Using the Margin Line Tool



The **Margin Line Tool** enables you to trace margin lines on 3D models, in order to ensure the accuracy of restorations ordered from the lab.

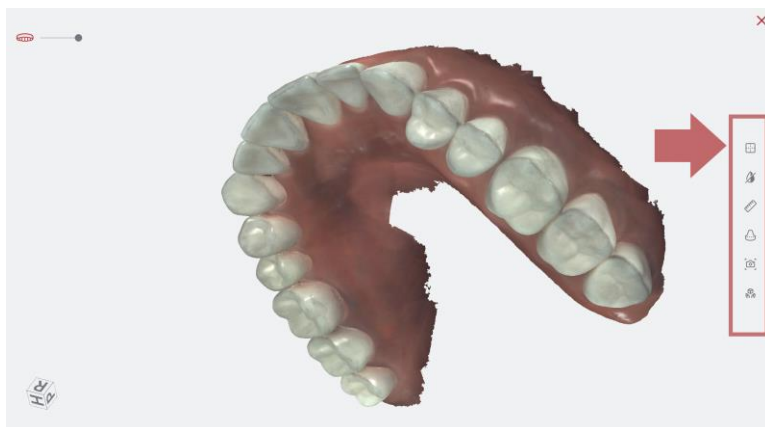

To access the **Margin Line Tool**, in the **Finalization Page**:

- STEP 1.** Click to expand the **3D Models** tab to full-screen.
- STEP 2.** Click on the **Margin Line Tool icon** in the right-hand **Case Finalization Tools** menu.


This opens the **Margin Line Tool : Step 1 - Select Tooth**

**Step 1 - Select Tooth**  
Select tooth on the teeth chart



### STEP 1. Select Tooth


- Click on the maxillary or **mandibular** icon to display the arch.
- Click to select a tooth already selected for restoration in **Case Setup**.
- Click **Next**.

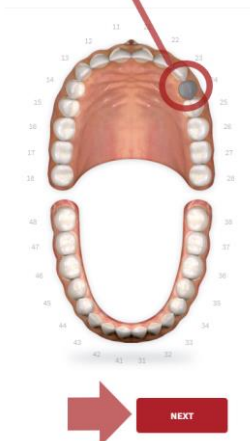

This opens the **Margin Line Tool : Step 2 – Highlight Tooth**

### STEP 2. Highlight Tooth

- To highlight a tooth/zone: **Click-left**
- To expand zone size: **Click-left** again (repeat as necessary).
- To adjust the zone size: **HOLD[CTRL] + scroll wheel**
- Click **Next**.

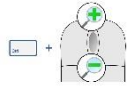

### STEP 3. Draw Margin Line:



- To start margin line: **Click-left** then trace line.
- To add control point: **Click-left** where required.
- Reposition 3D Model: **Click-left + drag**.
- Insert control point: **Click-left on line**.
- Finish margin line: **Double-Click-left**.
- Click **NEXT**.

The **Margin Line** can be viewed in the **3D MODELS** tab of the **Finalization** page.

## Modifying a Margin Line ¶

To modify a Margin Line:

1. Click to expand the **3D Models** tab to full-screen.
2. Click on the **Margin Line** icon in the right-hand **Case Finalization Tools** menu.
3. Click to select the tooth for which a margin Line has been drawn.
4. In the **Confirm** Dialog Box, to **Edit** the existing margin line click **EDIT**.

5. To delete the Margin Line, click **DELETE**.
6. Follow the **Margin Line** procedure described above.

## 8.5 Using the Model Closing Tool ¶




The **Model Closing** tool enables you to prepare 3D models for printing.

To access the Model Studio **Model Closing** tool, in the **Finalization Page**:

1. Click to expand the **3D Models** tab to full-screen.
2. Click on the **Model Studio** icon in the right-hand **Case Finalization Tools** menu.


This will open the **Model Studio** and invite you to prepare the digital 3D model for printing (also referred to as *closing* the model).

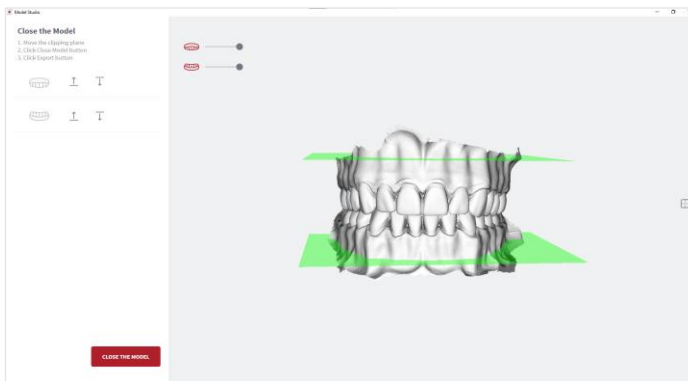

To **Close** the 3D Model:

1. Move the *clipping plane* to the desired position.
2. Click the button: **CLOSE THE MODEL**.



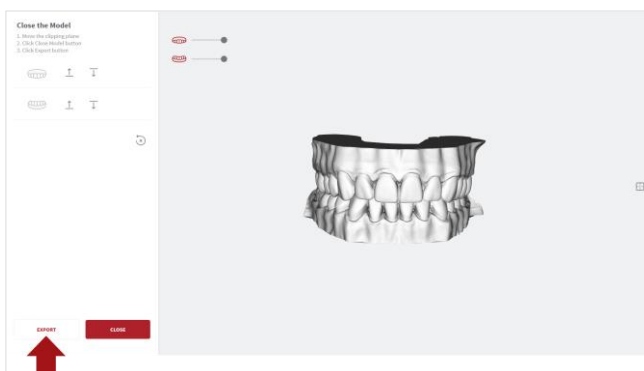

The closed 3D Model is displayed, and you are invited to **EXPORT** the model files.



3. Click **EXPORT**

4. Select or create an export folder, and Click OK.

The closed model files are exported in the **Case File Format** selected in the **Case Finalization** page.

This PC > Desktop > Close-Model > 2022-09-06-000118					
Name	Status	Date modified	Type	Size	
 Mandibular.stl		9/27/2022 10:14 AM	exocad view	39,644 KB	
 Maxillary.stl		9/27/2022 10:14 AM	exocad view	55,945 KB	


For more information, visit: **Case Finalization Options ¶**

5. Click **Close** to exit the **Model Studio** page.

## 8.6 Case Review: Reviewing 2D images

You can consult 2D images taken during the scan by clicking on the **2D IMAGES** tab in the **Case Review** page:

- STEP 1.** Click on the 2D Models tab.
- STEP 2.** Click on a thumbnail image to preview the selected 2D image.
- STEP 3.** Click on the expand icon to view 2D images in more detail.

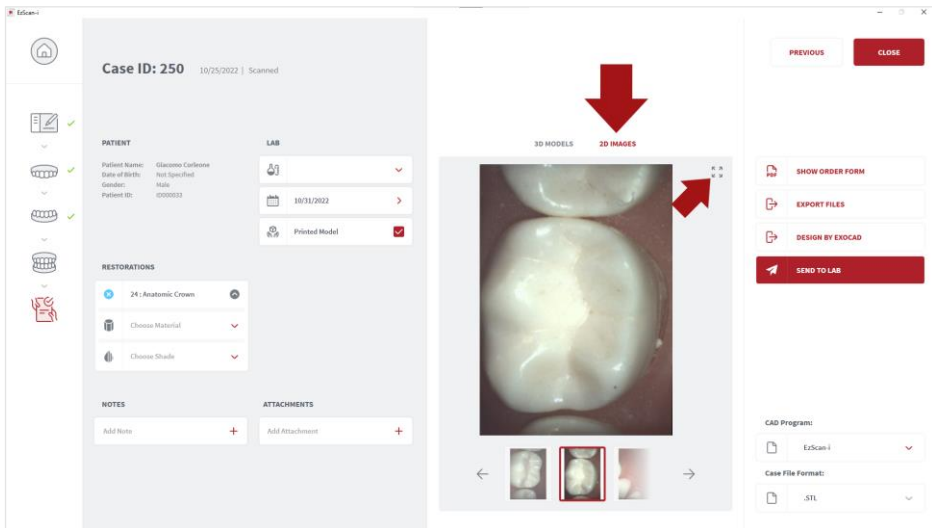

- STEP 4.** Delete unwanted images by selecting and clicking on the delete icon.

## 8.7 Case Review: Reviewing 3D Models

- STEP 5.** Click on the **3D Models** tab.
- STEP 6.** Expand full-screen to access all view and edit options.
- STEP 7.** Draw margin lines in **3D Model** as necessary.





## 8.8 Check And Send Your Order

### Verify Patient & Restoration details

- STEP 1.** Verify & complete as necessary **Patient** and **Restorations** information defined during the **Case Setup** step.
- STEP 2.** Add any notes and attachments as necessary.

### Verify Export settings

- STEP 1.** Verify/select **CAD Program**: (EzScan-i , Exocad, DWOS, 3OXS,...).
- STEP 2.** Verify/select **Case File Format**: (.STL, .OBJ, .PLY).
- STEP 3.** Verify/select Export (Local export, Exocad, L

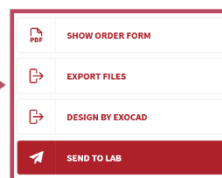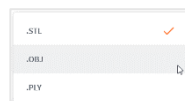

In **Data** settings, you can configure:


- **Case/Database Export** settings
- the **Exocad export** file path

NOTE: The **Design By Exocad** option will appear in **Case Finalization Export** options only when the Exocad export path has been configured in settings.

### Verify Lab Order Details

Verify and complete **Lab** order details:

- STEP 1.** Verify/select the desired lab connection displayed in the Case Preview **LAB** menu.
- STEP 2.** Enter or verify the desired delivery date (the connected lab's interactive calendar can display here to provide available dates).

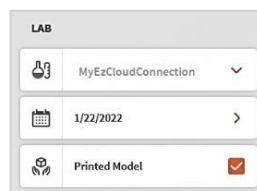

**STEP 3.** Tick the checkbox to select Printed Model if required.

## Select Default Lab



In **Upload** settings, you can add lab connections, and select a default lab from those connected to your EzScanCloud account.

The default lab will be displayed automatically in the **Case Setup** and **Case Finalization** pages. The case files and order form will be sent to this lab by default.

## Verify Order Form Details

**STEP 1.** Click **Show Order Form** to display any case **Order Form** (Lab Sheet PDF) tha to the lab.



### Lab Sheet details

In **Lab Sheet** settings, you can customize the prescription to be sent to the lab:

- Institution Name,
- Institution Address,
- Page Size,
- Institution Logo

**STEP 2.** Verify **Case**, **Patient** and **Order** details on the Lab Sheet.

- Verify **Restoration** information.
- Verify attached files and file format.

On page 2 of the **Lab Sheet**:

- Verify **3D Models** and **2D Photos**.


### STEP 3. Click Send To Lab.



This uploads the **Case files** and **Order Form** to the selected lab via the EzScanCloud connection configured for this lab.

The dental lab will receive an automatic email notification.

When the lab clicks on the email notification of your order, relevant Case details are displayed in their **Dental Lab** account in **EzScanCloud**.

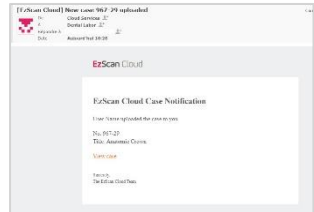

## Verify Case Status

To verify the status of **cases sent/resent** to labs:

### STEP 1. Click on the **status icon** in the **Case Review** page for details



See above: Case Status Icons

## Resending a Modified Order Form

If you wish to modify and resend an order to a lab, you will be asked if you want to replace the order sent or create a new order:

- If you choose to **replace** the case already sent to the lab, the same Case ID will be assigned in EzScanCloud.
- If you choose to **create a new case**, a new **Case ID** will be assigned in EzScanCloud.



**Note:** Before selecting a lab, you must first set up a connection with the lab via the **EzScanCloud** platform.

For more information, see:

- [section 4.6 - Linking a User Account to the EzScanCloud](#)
- [section 9.1 - Linking an EzScan-i account to EzScanCloud](#)

# 9. Communicating With Labs

The EzScanCloud enables you to send orders directly to labs, manage the progress and status of orders, and create and manage groups of Dental Clinic and Dental lab profiles.

To send orders to a lab via the EzScanCloud platform:

- STEP 1.** Link your EzScan-i account to the EzScanCloud
- STEP 2.** Create an EzScanCloud User Account
- STEP 3.** Configure a connection with the lab in EzScanCloud

Note: In EzScan-i, you can also communicate directly with labs by exporting files to your computer and transferring them via a third party platform.




**IMPORTANT:** The EzScanCloud platform is designed to facilitate file sharing and manage connections with laboratories. It is **NOT** intended as a cloud storage service.


## 9.1 Linking an EzScan-i account to EzScanCloud

To link your EzScan-i account to the EzScanCloud :

- STEP 1.** Click on the System Settings icon in the EzScan-i Start screen.
- STEP 2.** In **User** system settings, click **Add or Edit Users** using the administrator account (**HC**).
- STEP 3.** Select Use EzScanCloud in the Add or Edit Users dialog box.
- STEP 4.** Click **Save**.

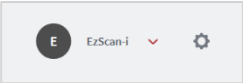

The **EzScanCloud Login** window will open automatically, inviting you to **Login** or to **Create a New Account**.


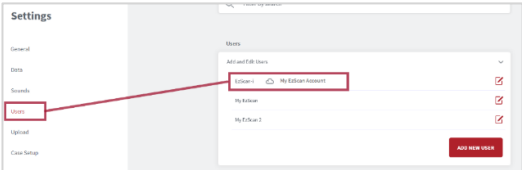


## 9.2 Creating An

# EzScanCloud User Account (Clinic)


To create a user account in **EzScanCloud**:

- STEP 1.** Click Create New Account
- STEP 2.** In the dialog box **fill** in the required fields (marked with a red \*)
- STEP 3.** Click **Create**.

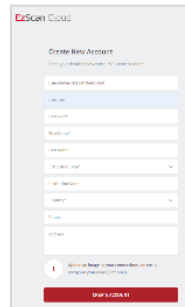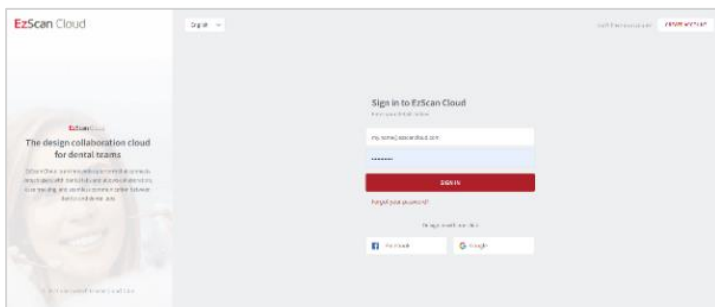

An email will be sent to confirm the email address.

- STEP 4.** Click the confirmation link to activate your **EzScanCloud** account.
- STEP 5.** Click **HOME** to access the **EzScanCloud** Sign In page.
- STEP 6.** Select a **language** in the scroll down menu.

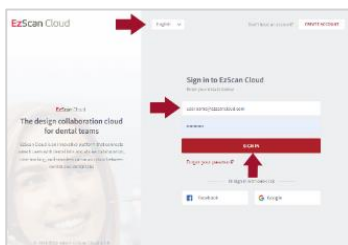


- STEP 7.** Enter your account email address and password.
- STEP 8.** Click **Sign In** to access your **EzScanCloud** Clinic account.

The **EzScanCloud** platform provides the lab with secure access to all Patient Case files associated with your Order.

## 9.3 Overview of the EzScanCloud Dashboard

The **EzScanCloud™ Dashboard** provides an at-a-glance overview of activity and options associated with your account:

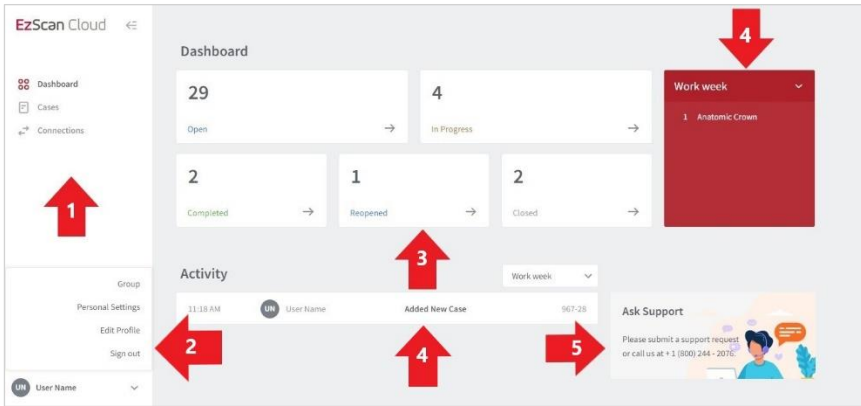

The **Dashboard** enables users to:

1. Access Patient Cases and configure connections with labs.
2. Configure account settings: Groups, Personal Settings, User Profile, Login/out.
3. Overview at-a-glance and access Patient Cases based on Case Status.
4. Select activity based on the display period.
5. Contact Support.



**IMPORTANT:** The EzScanCloud platform is designed to facilitate file sharing and manage connections with laboratories. **It is NOT intended as a cloud storage service.**


## 9.4 Configuring EzScanCloud User Settings

To configure user settings in EzScanCloud:

- STEP 1.

In the bottom left-hand menu, click on your account ID.
- STEP 2.

In the expanded list, select the desired option.

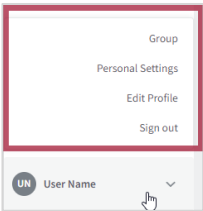

The following options are available in the scroll menu:

EzScanCloud Settings	Description
Edit Profile	Opens the Profile page, where you can enter and edit your EzScanCloud account details. Also enables you to access the Change Password window.
Sign out	To sign out from your account.
Group	Enables you to setup and manage Shared Groups.
Personal Settings	Enables you to configure view settings and notification settings for your EzScanCloud account.

## 9.5 Editing your EzScanCloud Profile

To edit your EzScanCloud profile:

- STEP 1.


Select **Edit Profile** in the expanded list.
- STEP 2.

In the **Profile** page, edit profile details.

### Dental Clinic and Dental Lab profiles

The EzScanCloud provides permissions and workflows specifically adapted to Dental Clinic and Dental Lab accounts.

**IMPORTANT:** Make sure you specify the correct **Dental Clinic** or **Dental Lab** profile in user settings.

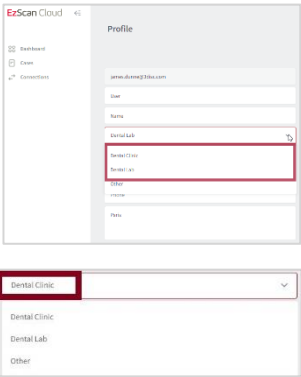



# 9.6 Configuring View & Notification Settings


To configure **view & notification settings**, in the **EzScanCloud Profile** expanded list:

- STEP 1.

Click Personal Settings.
- STEP 2.

In the Personal Settings page: select **View** and **Notification** settings for your **EzScanCloud** account.
- STEP 3.

Click Save Settings.

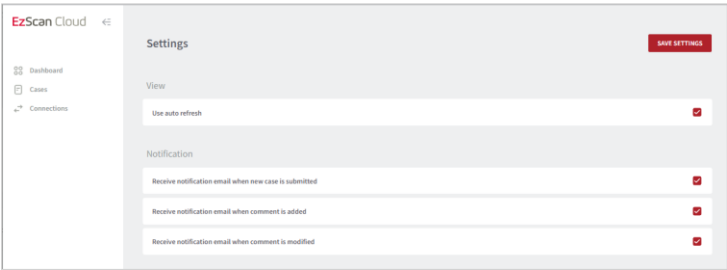

The following **view & notification settings** are available in **EzScanCloud**:

Settings	Description
View Settings	Show 'Cases/Drafts' filter above case list
	Show 'Assignee' column in case list
	Use auto refresh
Push Notification Settings	Receive notification email when new case is published
	Receive notification email when case is modified
	Receive notification email when comment is added
	Receive notification email when comment is modified

## 9.7 Configuring Connections with Labs



When you have created an EzScanCloud account and are logged in, you can now configure connections with the labs of your choice.

### Adding a lab connection

To add a connection with a lab:

**STEP 1.** Click **Connections** in the EzScanCloud menu.


**STEP 2.** Click Create New Connection.

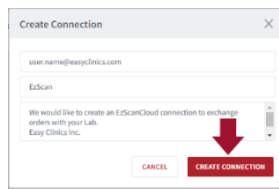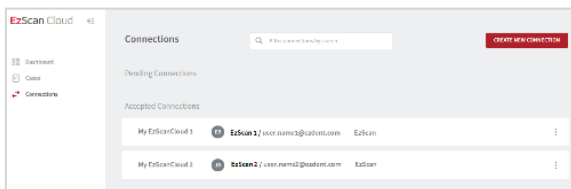

**STEP 3.** In the **Create Connection** dialog box, enter the Lab email address, Alias (optional) and Message.

**STEP 4.** Click **Create Connection**. A request email is sent to the lab inviting it to accept the connection.

**STEP 5.** Click **Connections** to view connection status or to add another lab connection.

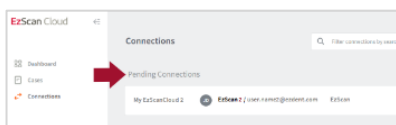

Current connection requests are displayed in your list of **Pending Connections** in **EzScanCloud**.

## Selecting Default Upload Settings in EzScan-i


you can select a **default lab connection** EzScan-i, to transfer files to via your **EzScanCloud** account:

**STEP 1.** Click the EzScan-i **Settings** icon.

**STEP 2.** Scroll to Upload Settings.

**STEP 3.** Click **Select Default Lab**, and choose a lab from the expanded list.

**STEP 4.** **Restart** the application to apply changes.

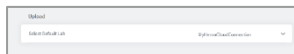

The **default Lab connection** will be selected automatically in the **Case Setup** and **Case Review** pages.

## 9.8 Managing Cases in EzScanCloud


To access patient cases: click on the **Cases** icon in the left-hand menu of the **EzScanCloud Dashboard**.



**STEP 1.** Click on the **Cases** icon in the left-hand menu.

The Cases page is displayed, showing the list of uploaded cases

**STEP 2.** Click on a case to display details.

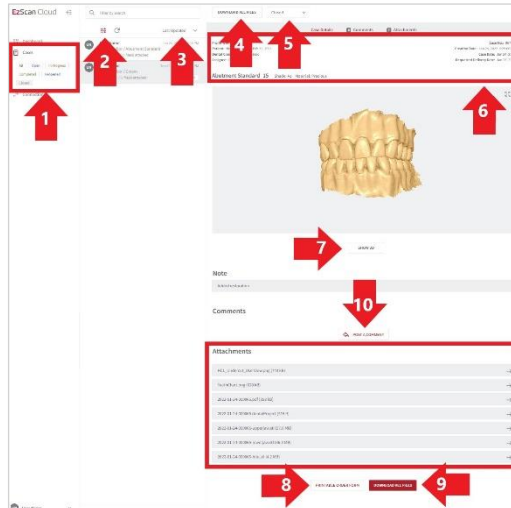

The **EzScanCloud Cases** page enables users to:



1. Select cases based on status.
2. Filter cases.
3. Display cases based on Last Updated, Creation date and/or Status.
4. Download all files associated with a selected case.
5. Modify the Case Status.
6. Consult Case details.
7. Display 3D file associated with a case.
8. Display the Case Order Form in PDF format.
9. Download all files associated with a case.
10. Add a comment or file to a case.

## 9.9 Managing Case Status in EzScanCloud

### Displaying Cases by Status

To display patient cases by status:


- STEP 1.** Click on the **Cases** icon in the left-hand menu
- STEP 2.** Click on a Status button to display the corresponding list of cases.
- STEP 3.** Click again on a Status button to deselect that status.  
(You can select multiple status).

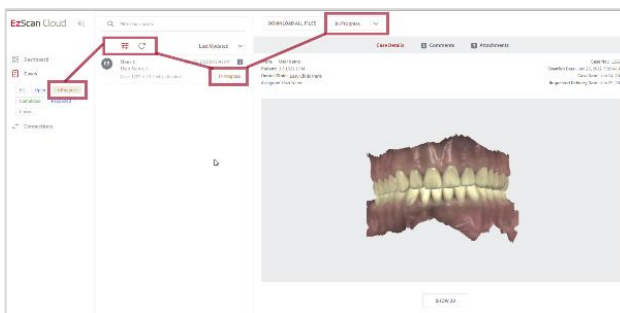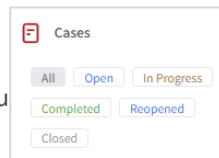

### Filtering Cases

To filter cases:

- STEP 1.** Select the filter icon at the top of your list of cases.
- STEP 2.** Select the status, creation period, and/or an institution/user.
- STEP 3.** Click **Apply**.  
To remove an active filter click **Reset All**.

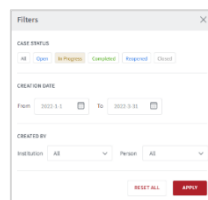

# Changing Case Status

To change the status of a case:

- STEP 1.


Click on the status in case details
- STEP 2.

Select a status from the expanded list.



Permissions enabling a user to modify the case status depend on whether the user profile is a Clinic or a Lab. Permitted options are displayed automatically.

## Case Status Descriptions

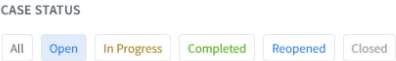

The following case status are available in EzScanCloud:

Status	Description	Managed by
Open	Case uploaded by Clinic to EzScanCloud	Clinic
In Progress	Case being processed by Lab (case status modified by lab)	Lab
Completed	Case treated and completed by Lab (case status modified by lab)	Lab
Reopened	Case closed and reopened. (case status modified by Clinic)	Clinic
Closed	Case closed. (case status modified by Clinic)	Clinic

## 9.10 Displaying 3D Views in EzScanCloud


To display the **3D View** for a selected case:

**STEP 1.** Click **SHOW 3D**.



**STEP 2.** To view the 3D model in full screen: click on the icon in the top right-hand corner.

**STEP 3.** To increase/decrease the transparency for an arch: drag the slider left/right.

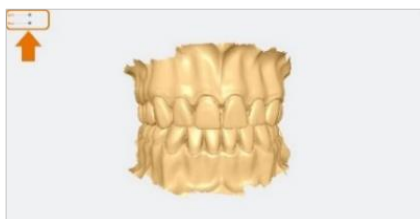

**STEP 4.** To hide/display an arch: click on the corresponding icon.

For information on configuring **3D Projection Settings**, see above:

- **Section 5.9 - Error! Reference source not found.: Error! Reference source not found.**

## 9.11 Downloading Case Files in EzScanCloud

### Downloading specific case files

To download specific case files, in the Case details page:

**STEP 1.** Scroll down to **Attachments**.

**STEP 2.** Click on the arrow icon opposite the file attachment.



### Downloading all case files








To download all files for a selected case:

**STEP 3.** click on the **Download All Files** button.

### Case File Details

Clicking the **Download All Files** button downloads a compressed (.zip) file containing the following:

- The Case **Project** file.
- **3D Scan files** (STL, OBJ, PLY) in the **Preferred Output Format** configured in the **Settings** page
- The case **Order Form** (PDF).

Name	Type	Compressed size	Password ...	Size	Ratio
 2022-05-25-000005.pdf	Adobe Acrobat Document	169 KB	No	180 KB	7%
 2022-05-25-000005.qcProject	QCPROJECT File	1 KB	No	2 KB	54%
 2022-05-25-000005-bite.stl	STL File	2,104 KB	No	3,153 KB	34%
 2022-05-25-000005-bite-opposing.stl	STL File	2,700 KB	No	3,936 KB	32%
 2022-05-25-000005-mandibular.stl	STL File	32,803 KB	No	49,671 KB	34%
 2022-05-25-000005-maxillary.stl	STL File	40,461 KB	No	61,683 KB	35%
 TeethChart.png	PNG File	619 KB	No	620 KB	1%

**NOTE:** To configure the **Preferred Output Format** for your case files, see above:

- **Section 5.1 - Error! Reference source not found..**

## 9.12 Displaying a Case Order Form in EzScanCloud

To display a **Case Lab Sheet/Order Form** in **EzScanCloud**:

**STEP 1.** Click the **Printable Order Form** button at the bottom of the **Case details** page.

PRINTABLE ORDER FORM

This will open the **Lab Sheet/Order Form** in .pdf format.

### Configuring Lab Sheet/Order Form settings

For information on configuring **Lab Sheet/Order Form settings**, see above:

### Section 0 -



- Lab Sheet Settings

## 9.13 Adding a Comment to a Case

To add a comment to a case in **EzScanCloud™**, in the **Case details** page:

**STEP 1.** Click on Post A Comment.

This will open the **Add your comment** dialog box.



**STEP 2.** Enter your comment and add any files as required.

**STEP 3.** Click Save Comment.

The lab to which you sent the case will receive an email notification to view your comment.



**NOTE:** You can also add comments to a case before upload, in the EzScan-i **Case Review** page.


For more information, see above:

- Chapter **Error! Reference source not found. - Error! Reference source not found.**

## 9.14 Contacting Support via EzScanCloud

To contact **Vatech Support** via the **EzScanCloud Dashboard**:

**STEP 1.** Click on **Ask Support** in the EzScanCloud Dashboard.

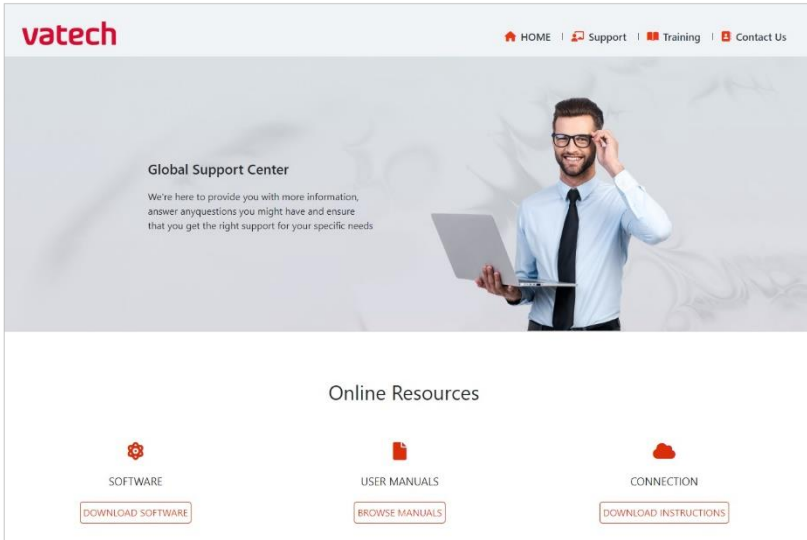

This opens the Support page at [EzScan \(vatechezscan.com\)](https://vatechezscan.com)

**STEP 2.** Follow Online Instructions.

# 10. Maintenance

## 10.1 Cleaning the Handpiece

The entire body, cord and base of the scanner must be wiped down using a Federal Environmental Protection Agency (EPA) approved disinfectant that is labeled and specified for tuberculocidal/ mycobactericidal activity. Do not use disinfectant on the nozzle.



**NOTE:** All components of the scanner (excluding the tips) must be wiped down and not sprayed. Avoid getting any moisture, alcohol or disinfectant inside the open scanner chamber.

### Recommended and approved surface disinfectants:

- Birex Wipes: TB Claim = 10 minutes- 'Phenolic (Dual) Water-Based'
- Prospray Wipes: TB Claim = 10 minutes- 'Phenolic (Dual) Water-Based'
- Cavicide Wipes: TB Claim = 3 minutes- 'Phenolics (Dual) Alcohol-Based'
- DisCide ULTRA Wipes: TB Claim = 1 minute- 'Phenolics (Dual) Alcohol-Based'
- Maxiwipe Germicidal Cloth: TB Claim = 5 minutes- 'Phenolics (Dual) Alcohol-Based'
- Ster 1 Plus: TB Claim = 5 minutes- 'Quaternary ammonium and Alcohol-Based'

## 10.2 Cleaning and Sterilizing Tips





**NOTE:** The included tips must be autoclaved prior to use as they do not come pre-sterilized.



**NOTE:** Ensure that the surface of the mirror does not show residues, smudges, scratches, or any damage, as this would affect the performance of the device.

## Procedure

- STEP 1.** Clean the tip with soapy water, ensuring that the mirror is clean and free of smudges, stains, or any residue. Avoid using abrasive cloth materials as this will scratch the mirror.
- STEP 2.** After drying the tip exterior, carefully dry the interior and mirror with lint free wipes ensuring you do not scratch the surface. The mirror should be free of any noticeable debris or water spots.
- STEP 3.** Insert and seal the tip into a sterilization pouch. Make sure the seal is airtight. Each tip should be packaged individually.
- STEP 4.** Sterilize the wrapped tip in a steam autoclave at following parameters:
- 132°C (270°F) at 4 minutes, or
  - 134°C (273°F) at 4 minutes, or
  - 121°C (250°F) at 45 minutes
- STEP 5.** Ensure the dry cycle is complete prior to removing the tip from the autoclave. If the pouch is damp with moisture, proper sterilization cannot be guaranteed.

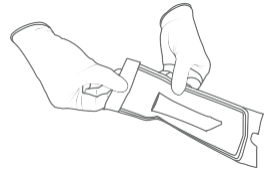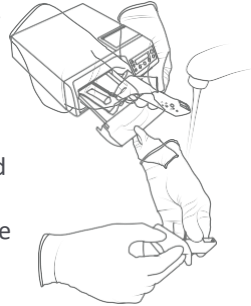

**WARNING:** Always autoclave the tip wrapped in a sealed sterilization pouch; failure to do so will result in permanent stains on the mirror.



**NOTE:** Tips should not be placed in an ultrasonic cleaner or any cold sterile solutions. The sterilant solutions will leave a sticky residue or film on the mirror when drying.



**WARNING:** Do not autoclave the handpiece of the device.



**WARNING:** Do not remove the pouch before the sterilizer completes its full dry cycle. If the pouch is wet or has any signs of moisture, this can potentially leave water spots on the mirror which can affect image quality during scanning.



**WARNING:** Use extreme caution when cleaning the mirror as it is very delicate and is prone to scratching.

## 10.3 Disposal

The EzScan is an electrical device with electronical components inside and should be disposed of in accordance with local environmental laws and regulations.

## 10.4 Calibration

The EzScan is calibrated in the factory and therefore does not require calibration when installed.



**WARNING:** General prohibition indication. The functionality of the system can be destroyed in the case of incorrect use. If unauthorized changes have been made to the delivered system and accessories, the warranty by Vatech becomes void. Vatech will not accept any responsibility or liability for the improper functioning of the product in such a case.

If the EzScan begins to have problems scanning and recognizing teeth models, contact your dealer or Vatech support technician.

If the scanner cannot be recalibrated remotely, this may result in the system being returned for repair/ calibration.

For more information see: [Chapter 12 - Support, Warranty & Repair Service.](#)

# 11. Safety Guidelines and Warnings

## 11.1 Warnings and Symbols



**NOTE:** Notes represent information that is important to know but which do not affect the functionality of the system.



**WARNING:** The functionality of the system will be limited in the case of incorrect use.

## 11.2 General Guidelines

- Do not spill liquids on the body of the device
- Never operate the device in a wet environment.
- Keep the device away from radiators and heat sources.
- Use the device only with the accessories supplied.
- Do not alter the device or open enclosures.



**WARNING:** General prohibition indication. The functionality of the system can be destroyed in the case of incorrect use. If unauthorized changes have been made to the delivered system and accessories, the warranty by Vatech becomes void. Vatech will not accept any responsibility or liability for the improper functioning of the product in such a case.

If any of the following conditions occur, unplug the device from the electrical outlet and contact authorized service personnel:

- The power cord or power adapter is damaged.
- The device has been exposed to water.
- The device has been damaged.
- The device does not operate correctly when the operating instructions are followed.

## 11.3 General Warnings

### System Modification



**WARNING:** Modifying the system may result in physical injury to the patient and operator, and damage to the system.

### Approved Software

The EzScan device is designed to operate with the EzScan-i software.



**WARNING:** The EzScan scanner should only be used with approved, compatible software.

### Equipment Failure



**WARNING:** In case of system malfunction or failure, you should:  
Prevent any contact between the system and the patient.  
Unplug the system from the power outlet and the computer.  
Store the system away so it cannot be used by someone else.  
Contact service personal.

## 11.4 Mechanical Hazards

### Moving Parts



**NOTE:** All moving parts are inside handheld scanner so do not open the unit.

## Dropped Equipment



**WARNING:** If the scanner tip is dropped, ensure that the mirror is not damaged and that it is not detached; if the tip is damaged it should be disposed of immediately. If the scanner handpiece is dropped or bumped, ensure that no part of the system is damaged as it could affect performance.



**NOTE:** When not in use, always rest the handpiece on the Base. The Base may be mounted on the wall per provided instructions. Do not place the Base on a slanted surface. Place the cables (power cable and USB cable) where people cannot accidentally get caught in them and potentially damage the system.

## Base

### 11.5 Electrical Safety

## Electrical Shock



**WARNING:** There is a risk of electrical shock when opening or attempting to open any part of the system; only qualified service personal should open parts of the system.

## Stress on Cables



**WARNING:** Care should be taken not to apply unnecessary stress on the cables of the system, whether it is the power cable, the USB cable or the cable between the handpiece and the Base.



## 11.6 Eye Safety



**WARNING:** During operation, the system emits a bright, flashing light from its tip. Although the system complies with standard IEC 62471 (Photobiological safety of lamps and lamp systems), prolonged exposure to flashing light may result in discomfort, seizure or eye irritation.



**WARNING:** Only use the power adapter supplied as a part of the system.

## 11.7 Hygiene

**WARNING:** In order to maintain safety for the patient, wear surgical gloves when handling any parts of the system. Always ensure that the tip is mounted on the handpiece before inserting it into the mouth of the patient. Before using the system with a new patient, ensure that the system is disinfected, and the tip is sterilized.

## 11.8 Precautions During Systems Operation



**WARNING:** The EzScan system contains delicate optical and mechanical elements and therefore should be handled carefully. Do not drop, bump, or shake the handpiece or the tip. Always place the handpiece on the Base when not in use. Do not put stress on the cable connecting the handpiece to the Base. Do not submerge the handpiece or the Base in any liquid. Do not place the handpiece or the Base on wet or heated surfaces. Hold the handpiece with a firm grip when handling it.



**WARNING:** Portable RF communication equipment (including peripherals such as antenna cables and external antennas) should not be used within 30 cm of any part of the EzScan, including cables specified by the MANUFACTURER. Otherwise, performance degradation of this equipment may occur.



**NOTE:** During operation of the system, the handpiece and the tip may get slightly warm; this is normal.



**WARNING:** In order to prevent over-heating of the system, the ventilation opening at the bottom of the handpiece should never be obstructed.



**WARNING:** Use of this equipment adjacent to or on other equipment must be avoided as it may result in improper operation. If this use is necessary, it is advisable that this and the other equipment be observed to verify that they are operating normally.

## 11.9 EzScan Performance



**WARNING:** Under normal use, the EzScan should transmit images to the laptop/notebook with the installed image manipulation software and the video stream is visible in the bottom left corner of the screen. If not used properly, there may be loss of transmission of image information or slow transmission temporarily.

# 11.10 Accessories



**WARNING:** Only use Vatech approved accessories. Not using approved accessories may result in deterioration of performance.

Accessory	Vatech Part Number
USB 3.0 Cable	IOS-CP-00-043
AC/DC Power Adapter	IOS-CP-00-088

EzScan is intended for use in a professional healthcare setting with electromagnetic environment specified below.


# 11.11 EMC Guidance and Declaration

## Electromagnetic Emissions

Emissions Test	Compliance	Electromagnetic Environment Guidance
RF Emissions CISPR 11	Group 1, Class A	EzScan uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
Harmonic emissions IEC 61000-3-2	Class A	
Voltage Fluctuations/Flicker Emissions IEC 61000-3-3	Complies	EzScan is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.

## Electromagnetic Immunity

Immunity Test	Test Level	Compliance Level	Electromagnetic Environment Guidance
<b>Electrostatic discharge (ESD) IEC 61000-4-2</b>	± 8 kV /Contact ± 2 kV, ± 4 kV, ± 8 kV, ± 15 kV /air	± 8 kV /Contact ± 2 kV, ± 4 kV, ± 8 kV, ± 15 kV /air	Surface should be wood, concrete, or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%.
<b>Electrical fast transient/ burst IEC 61000-4-4</b>	± 2 kV for power supply lines ± 1 kV for input/output lines	± 2 kV for power supply lines ± 1 kV for input/output lines	Mains power quality should be that of a typical professional healthcare facility environment.
<b>Surge IEC 61000-4-5</b>	± 0.5 kV, ± 1 kV line(s) to line(s) ± 0.5 kV, ± 1 kV, ± 2 kV line(s) to earth	± 0.5 kV, ± 1 kV line(s) to line(s) ± 0.5 kV, ± 1 kV, ± 2 kV line(s) to earth	Mains power quality should be that of a typical professional healthcare facility environment.
<b>Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11</b>	<5% UT (>95% dip in UT) for 0,5 cycle 40% UT (60% dip in UT) for 5 cycles 70 % UT (30% dip in UT) for 25 cycles <5% UT (>95% dip in UT) for 5 s	<5% UT (>95% dip in UT) for 0,5 cycle 40% UT (60% dip in UT) for 5 cycles 70 % UT (30% dip in UT) for 25 cycles <5% UT (>95% dip in UT) for 5 s	Mains power quality should be that of a typical professional healthcare facility environment.
<b>Power frequency (50/60 Hz) magnetic field IEC 61000-4-8</b>	30 A/m	30 A/m	Power frequency magnetic fields should be at levels characteristic of a location in a typical professional healthcare facility environment.

<b>Radiated RF IEC 61000-4-3</b>	Table 9 in IEC-60601- 1-2 2014	Table 9 in IEC-60601- 1-2 2014	<p>Portable and mobile RF communications equipment should be used no closer to any part of the EzScan system, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.</p> <p>Recommended separation distance:</p> $d = \left\lceil \frac{3,5}{\sqrt{f}} \right\rceil \sqrt{P} \quad 150 \text{ kHz to } 80 \text{ MHz}$ $d = \left\lceil \frac{3,5}{E1} \right\rceil \sqrt{P} \quad 80 \text{ MHz to } 800 \text{ MHz}$ $d = \left\lceil \frac{7}{E1} \right\rceil \sqrt{P} \quad 800 \text{ MHz to } 2.5 \text{ GHz}$ <p>where P is the maximum output power rating of the transmitter in watts (W) according to the manufacturer and d is the recommended separation distance in metres (m). Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey, should be less than the compliance level in each frequency range. Interference may occur in the vicinity of equipment marked with the following symbol:</p> 
<b>Conducted RF IEC 61000-4-6</b>	3V 0.15-80MHz 6V in ISM bands between 0.15 MHz and 80 MHz 80% AM at 1KHz	3V 0.15-80MHz 6V in ISM bands between 0.15 MHz and 80 MHz 80% AM at 1KHz	


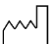







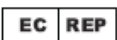








*NOTE: UT is the A.C. mains voltage prior to application of the test level.*

## **Recommended Separation Distances Between Portable and Mobile RF Communications Equipment and the EzScan System that is not Life-Supporting**




EzScan System is intended for use in the electromagnetic environment in which radiated RF disturbances are controlled. The customer can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the EzScan system as recommended below, according to the maximum output power of the communications equipment.

Rated maximum output power of transmitter W	Separation distance according to frequency of transmitter m		
	150 kHz to 80 MHz $d = \left[\frac{3.5}{V_1}\right]\sqrt{P}$	80 MHz to 800 MHz $d = \left[\frac{3.5}{E_1}\right]\sqrt{P}$	800 MHz to 2.5 GHz $d = \left[\frac{7}{E_1}\right]\sqrt{P}$
0,01	0.12	0.12	0.23
0,01	0.38	0.38	0.73
1	1.2	1.2	2.3
10	3.8	3.8	7.3
100	12	12	23
<p>For transmitters rated at a maximum output power not listed above, the recommended separation distance d in metres (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.</p> <p><i>NOTE 1: At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.</i></p> <p><i>NOTE 2: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.</i></p>			

## 11.12 Symbols on the EzScan and TransportCase

Symbol	Description
	Manufacturer's trade name and address (ISO 15223-1)
	Date of manufacture (ISO 15223-1)
	Equipment Power On/Off (push/push)
	USB plug
	Warning, Consult Accompanying Documents
	General mandatory action manual
	General prohibition indication
	User Manual Reference
	Directive on Waste Electrical and Electronic Equipment
	Authorized Representative in the European Community
	Warning label for LED
	Non-ionizing electromagnetic radiation
	Direct Current
	Type(B) Level of protection against electric shock
	Consult operating instruction for use.
	European Conformity mark
	Prescription symbol
	Non-sterile (Scanner-Tips) (IOS-FPL-71-001)



	User manuals are available electronically at the link provided: <a href="http://vatechezscan.com/support">http://vatechezscan.com/support</a>
	INMETRO Certification Mark
	Ukraine Conformity Mark

## 12. Support, Warranty & Repair Service

### 12.1 Support

If you have questions about the software, please consult the manual and Help menu in the software. If you are experiencing issues with your software, please check the list of common issues provided below prior to contacting a dealer. It could be simply a question of a minor issue that can be fixed quickly. However, if you're still experiencing problems after following the recommendations in this section, then please contact the dealer where you bought the equipment.

#### Operating Issues Checklist

Issue	Recommendation
There's a memory full error message that pops up when the software is open.	Clear some space on the C Drive
The status in the Live view window is "Disconnected".	Check that you have external power to the EzScan and that the USB cable is connected to a USB 3 Port.
Scanning is very slow.	Check that the Laptop is connected to an external power source.
The corners are cut in the live view window.	Check that the Tip is correctly mounted and when rotating it is locking in place with a click.
There is a red square in the scan window	Go back to a tooth that is scanned and start from there again
No images appear when scanned but everything else (e.g. live window image, sounds, FPS) works fine.	The scanner might need to be recalibrated. Please contact your local dealer for support.
There are spots on the Live view window.	Check and clean the mirror of the tip.
Where can I get the EzScan software and manuals?	Please find them in the Support section of Vatech website.

### 12.2 Standard Warranty

Vatech warrants its non-consumable hardware products to be free from defects in materials and workmanship. The warranty covers the cost of parts and labor to repair the product.

Please keep the shipping container for future use. Products returned to the factory for repair should be properly packaged. To obtain warranty service, follow the procedure described in the Repair Service section. Failure to do so will cause delays and additional expense to the customer.

The warranty is valid when the product is used for its intended purpose and does not cover products which have been modified without written permission from 3D Imaging and Simulation Corp. Americas, or which have been damaged by abuse, accident or connection to incompatible equipment.

This warranty is in lieu of all other warranties, expressed or implied.

## 12.3 Repair Service

The EzScan cannot be serviced locally. In the event of a hardware malfunction, contact your dealer to arrange for a swap unit (same model or newer) so your unit can be replaced, and work can continue. Some testing might be needed in order to verify the Hardware/Software error or malfunction.

The company reserves the right to cease providing repair, maintenance, parts and technical support for its non-consumable hardware products five years after a product is discontinued.

## 12.4 Out of Warranty Repair Service

Out of warranty repair service is available in selected geographical locations. Contact the supplier for current terms and rates.

We hope this User Manual was helpful to you.  
For additional material and user information go to  
**[www.VatechEzScan.com/support](http://www.VatechEzScan.com/support)**

EzScan User Manuals  
EzScan How-to Videos  
EzScan Training Videos


**[www.VatechEzScan.com/support](http://www.VatechEzScan.com/support)**

**Distributed by:**

Vatech America  
2200 Fletcher Ave.  
Suite 705A  
Fort Lee, NJ 07024  
USA

Toll Free: 1.888.396.6872

[www.VatechEzScan.com](http://www.VatechEzScan.com)

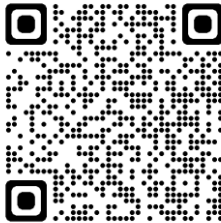

IOS-FPM-71-007\_EN\_AA