

Green CT 2 – 2D Panorama Patient Instruction Guide

To Patient:

Please follow these instructions to capture a 2D Panorama Xray:

Patient Positioning

- 1) Hands** – Palms facing up on the bottom of the handlebar
- 2) Feet** – Heels aligned with column, feet shoulder width apart
- 3) Head** – Chin on chinrest, and align teeth in the grooves of the bite stick

Patient Instructions

- 1)** Swallow and position ENTIRE tongue to the roof of your mouth
- 2)** Breathe through your nose
- 3)** Close your eyes
- 4)** Remain still for 30 seconds



VATECH
ASSURANCE