



Vatech America, Inc.
 2200 Fletcher Ave, Ste 705A
 Fort Lee, NJ, 07024
 Tel) 201-210-5028, Fax) 201-601-0859
www.vatechamerica.com

Total CBCT File Size for Exports with and without a 3D Viewer and whether a CD and/or DVD will Suffice

FOV	CBCT File Size With Viewer	Voxel Size	CD/DVD	CBCT File Size Without Viewer	Voxel Size	CD/DVD
5x5	829 MB	.08 Voxel	DVD	482 MB	.08 Voxel	CD/DVD
8x9	970 MB	.12 Voxel	DVD	647 MB	.12 Voxel	CD/DVD
10x9 Smart	593 MB	.2 Voxel	CD/DVD	260 MB	.2 Voxel	CD/DVD
12x9	644 MB	.2 Voxel	CD/DVD	308 MB	.2 Voxel	CD/DVD
16x9	900 MB	.2 Voxel	DVD	550 MB	.2 Voxel	CD/DVD
18x10	1100 MB	.2 Voxel	DVD	780 MB	.2 Voxel	DVD
17x15	867MB	.2 Voxel	DVD	517 MB	.2 Voxel	CD/DVD
21x19	946 MB	.3 Voxel	DVD	593 MB	.3 Voxel	CD/DVD